

Notice Information: - Advisory
23 May 2011

Part 1. Product Information

- a) Title:
- b) Product Name/Type:
- c) Reference:
- d) Manufacturer/Supplier:

Part 2. Target Audience

- a) Target Audience:

Part 3. Problem/Issue

- a) Problem/Issue:

Part 4. Background Information

a) Background Information:

Zoll received reports that following a long period without use (typically greater than four years); a ZOLL® AED Plus® may prompt “change batteries” during use and fail to deliver therapy. Turning the device off completely, waiting ten seconds for the unit to re-set and then turning it back on is necessary to resume proper operation.

Zoll's investigation of this problem has determined that some batteries may develop high internal resistance that interferes with the batteries' expected performance, lengthening charging time beyond specified and clinically acceptable limits.

In February 2009, ZOLL Medical Corporation started to inform owners of all AED Plus devices with serial numbers below X_ _ _200000 to replace batteries that have been installed in devices for a period longer than three (3) years and at subsequent three (3) year intervals.

Units above serial number X_ _ _200000 are not affected by this corrective action as they contain software that can detect this battery condition and identify when batteries require replacement. In lieu of adding the battery replacement label, users may obtain this new software for their AEDs at no cost at www.ZOLLAEDPlusbatteryhelp.com

Four distributors have supplied the affected product in Ireland, Pulse Medical Ltd, Heartbeat Safety Ltd, AED Direct and Medilink Services. Zoll has been working with the distributors to ensure that the issue is communicated to all users of the product in Ireland. Zoll have been unable to confirm to the IMB that the recommended actions have been completed on all devices.

Part 5. Action to be taken

a) Action to be taken:

1. Ensure that the relevant personnel in your organisation are made aware of this

issue.

2. Determine if you have purchased the affected serial numbers of this product,

AED Plus devices with serial numbers below X_ _ _200000.

3. If you have an affected device confirm that the recommended actions have been completed:

Ensure that the date when the batteries were installed in your device has been checked.

a. If batteries were installed more than three (3) years ago, replace the batteries

as soon as possible and add the label provided by Zoll showing the next

scheduled replacement date. Read and add the addendum incorporating this

information to the administrator's guide that was provided by Zoll.

b. If batteries are not more than three (3) years old, add the label provided by

Zoll completed with the next scheduled replacement date. Read and add the

addendum, provided by Zoll, incorporating this information to the administrator's guide.

Part 6. Enquiries

a) All enquiries should be made to:

All adverse incidents relating to a medical device should be reported to the:

Irish Medicines Board
Kevin O'Malley House
Earlsfort Centre
Earlsfort Terrace
Dublin 2

Telephone: +353-1-6764971

Fax: +353-1-6344033

E-mail: vigilance@imb.ie

Website: www.imb.ie

Enquiries to the manufacturer should be addressed to:

Manufacturer:

Zoll Medical Corporation
269 Mill Road
Chelmsford
Massachusetts
USA

Telephone: + 31 651937017

E-mail: erozeboom@zoll.com

Distributor:

Pulse Medical Ltd

Unit 8 Ba

Distributor:

Pulse Medical Ltd

Unit 8

Ballyboggin Business Centre

Ballyboggan Road

Glasnevin

Dublin 11

Ireland

Telephone: +353 1 8602631

Fax: +353 1 8602640

Heartbeat Safety Ltd.

Unit 3

Crookstown Business Park

Crookstown

Co Kildare

Ireland

Telephone: +353 59 8623975

Fax: +353 59 8623870

Medilink Services Link House 81 Sydenham Road Belfast Northern Ireland

Medilink Services

Link House

81 Sydenham Road

Belfast

Northern Ireland

Telephone: +44 2890 582999

Fax: +44 2890 582900

[Please click here to download a PDF version of the Safety Notice](#)