

Urgent Field Safety Notice

Endo GIA™ Surgical Stapling Single Use Loading Units, Tri-Staple™ 2.0 Intelligent Reloads and Cartridges Recall

May 2019

Medtronic reference: FA868

Dear Healthcare Professional,

The purpose of this letter is to advise you that Medtronic is voluntarily recalling specific production lots of its Endo GIA™ Surgical Stapling Single Use Loading Units, Tri-Staple™ 2.0 Intelligent Reloads and Cartridges.

Issue Description:

This voluntary recall is being conducted due to the potential for a device to be missing one of two pin components that maintain alignment of the device jaws. The affected devices were distributed between April 2014 and April 2019. Use of a product without a pin may result in incomplete staple formation, potentially leading to bleeding, anastomotic leak, peritonitis, or pneumothorax which can result in the potential for infection and/or sepsis. This potential issue was identified during in-process quality testing at the manufacturing facility. Manufacturing process improvements have been implemented to address this issue. There have been no confirmed complaints received related to this issue.

This voluntary recall affects only the item codes and lots listed on Attachment A.

Required Actions:

1. Please immediately quarantine and discontinue use of affected item codes and lots listed on Attachment A.
2. Please return affected product as indicated below. All unused products from the affected item codes and lots must be returned.
3. If you have distributed the Endo GIA™ Surgical Stapling Single Use Loading Units, Tri-Staple™ 2.0 Intelligent Reloads and Cartridges listed on Attachment A, please promptly forward the information from this letter to those recipients.
4. If you have purchased these products from a distributor. You need to contact your distributor directly to arrange for return of product.


This notice needs to be passed on to all those who need to be aware within your organization or to any organization where the potentially affected devices have been transferred.

The Competent Authority of your country has been notified of this action. Please maintain a copy of this notice in your records.

Medtronic

We regret any inconvenience this may cause. We are committed to patient safety and appreciate your prompt attention to this matter. If you have any questions or concerns, please do not hesitate to contact your Medtronic representative at 01 511 1400.

Sincerely,

A handwritten signature in black ink that reads "S. Baxter". The signature is written in a cursive style with a horizontal line underlining the name.

Samantha Baxter
Regulatory Affairs Manager UK and Ireland

Enclosure: Attachment A: Affected Product List

Attachment A: Affected Product List

| Item | Description | Affected Lot | | | | | |
|-------------|---|--------------|-----------|-----------|-----------|-----------|------------|
| EGIA30AMT | Endo GIA™ 30 mm Articulating Medium/Thick Reload with Tri-Staple™ Technology | N5A0871KX | N7A0527KX | N7E0441KX | N8J0039KX | N8J0527KX | N8J0820KX |
| EGIA45CTAMT | Endo GIA™ 45 mm Curved Tip Articulating Medium/Thick Reload with Tri-Staple™ Technology | N8L0401KY | | | | | |
| EGIA60AXT | Endo GIA™ 60 mm Extra Thick Black Articulating Reload with Tri-Staple™ Technology | N6B0079KX | N8A0029KX | | | | |
| EGIA60CTAMT | Endo GIA™ 60 mm Curved Tip Articulating Medium/Thick Reload with Tri-Staple™ Technology | N8L0311KY | N8M0063KY | | | | |
| SIG30AMT | Tri-Staple™ 2.0 Intelligent Reload for use with Signia™ stapling system 30 mm Medium/Thick | N6E0033UX | N7A0018UX | N7C0238X | N8A0188X | N8E0092X | N8F0819X |
| | | N6H0772UX | N6L0687UX | N7C0470X | N8B0483X | N8E0516X | N8J0440X |
| | | N6H0768UX | N7A0213UX | N7E0496X | N8B0057X | N8E0213X | N8L0589Y |
| | | N6K0522UX | N7A0401UX | N7E1088X | N8B0940X | N8E1178X | N8M0529Y |
| | | N6K0521UX | N6M0399UX | N7F0670X | N8D0649X | N8F0483X | N9A0427Y |
| | | N6K0667UX | N7A0837UX | N7H0962X | N8D0365X | N8F0974X | N9A0050Y |
| | | N6K0668UX | N7B0787UX | N7J0370X | | | |
| SIG45AXT | Tri-Staple™ 2.0 Black Intelligent Reload for use with Signia™ stapling system 45 mm Extra Thick | N6E0278UX | N6F0142UX | N6F0340UX | N6G0035UX | N6J0436UX | N6K1110UX |
| | | N6E0176UX | N6F0144UX | N6F0436UX | N6G0036UX | N6K0010UX | N7C0499X |
| | | N6E0584UX | N6F0342UX | N6G0037UX | N6E0587UX | N6K0372UX | N7C0111X |
| | | N6E0589UX | N6F0343UX | N6G0038UX | N6G0125UX | N6K0415UX | N7C0611X |
| | | N6E0590UX | N6F0141UX | N6G0033UX | N6G0744UX | N6K0991UX | N7D0303X |
| | | N6E0975UX | N6F0143UX | N6G0034UX | N6H0757UX | N6K0990UX | N7C0381X |
| | | N6F0341UX | | | | | |
| SIG45CTAMT | Tri-Staple™ 2.0 Curved Tip Intelligent Reload for use with Signia™ stapling system 45 mm Medium/Thick | N6E0296UX | N6L0721UX | N7F1125X | N7M0203X | N8C0854X | N8E1009X |
| | | N6J1035UX | N7A0414UX | N7A0844UX | N7M0759X | N8D0508X | N8E1028X |
| | | N6J1036UX | N7B0401UX | N7H0540X | N8A0152X | N8D1062X | N8J0178X |
| | | N6J1034UX | N7C0194X | N7H1065X | N8A0977X | N8E0561X | N8M0424Y |
| | | N6L0007UX | N7E0781X | N7K0033X | N8A0404X | N8E0932X | N8M0530Y |
| | | N6L0365UX | N7F0551X | N7K0564X | N8B0943X | N8E0989X | N8M0531Y |
| | | N6M0608UX | N7F1021X | N7L0401X | N8C0165X | N8E1006X | |
| SIG60AXT | Tri-Staple™ 2.0 Black Intelligent Reload for use with Signia™ stapling system 60 mm Extra Thick | N6C0533UX | N6E0977UX | N6E0976UX | N6G0784UX | N6H0302UX | N6K0970UX |
| | | N6C0582UX | N6F0508UX | N6F0049UX | N6G0785UX | N6H0734UX | N6K1067UX |
| | | N6C0657UX | N6F0509UX | N6F0503UX | N6G0786UX | N6H0693UX | N6L0152UX |
| | | N6C0681UX | N6F0758UX | N6F0504UX | N6H0105UX | N6H0909UX | N6M0328UX |
| | | N6E0115UX | N6F0295UX | N6F0505UX | N6H0106UX | N6H0946UX | N7A0170UX |
| | | N6E0116UX | N6F0748UX | N6E0979UX | N6H0107UX | N6G0068UX | N7C0475X |
| | | N6E0978UX | N6F0759UX | N6F0507UX | N6H0173UX | N6J0092UX | N7C1091X |
| | | N6F0045UX | N6F0760UX | N6F0506UX | N6H0298UX | N6J0040UX | N7C1092X |
| | | N6F0046UX | N6F0786UX | N6G0270UX | N6H0299UX | N6G0486UX | N7D0104X |
| | | N6F0047UX | N6F0787UX | N6G0271UX | N6H0155UX | N6H0174UX | N8A0087X |
| | | N6F0048UX | N6F0788UX | N6G0287UX | N6H0156UX | N6K0302UX | N8D0035X |
| | | N6F0050UX | N6G0288UX | N6G0499UX | N6H0157UX | N6H0522UX | N8D0225X |
| SIG60CTAMT | Tri-Staple™ 2.0 Curved Tip Intelligent Reload for use with Signia™ stapling system 60 mm Medium/Thick | N6E0295UX | N7A0415UX | N7H0047X | N8A0655X | N8E0015X | N8K0823KY |
| | | N6H0576UX | N7C0112X | N7G0444X | N8B0455X | N8D0907X | N8M0845Y |
| | | N6J1092UX | N7D1013X | N7H1067X | N8B0945X | N8G0648X | N8L0166Y |
| | | N6K0230UX | N7E1089X | N7J0175X | N8C1213X | N8J0764X | N9A0204Y |
| | | N6K0938UX | N7F0275X | N7L0825X | N8C0757X | N8K0840Y | N9B0455Y |
| | | N6L0777UX | N7F1126X | N8A0249X | N8D0593X | N8L0436Y | N6L0422URX |
| N6M0607UX | | | | | | | |

Attachment A: Affected Product List

| Item | Description | Affected Lot Numbers | | | | | |
|-----------|---|----------------------|-----------|-----------|------------|-----------|-----------|
| EGIA45AMT | Endo GIA™ 45 mm Articulating Medium/Thick Reload with Tri-Staple™ Technology | N7L1098KX | N5M0172KX | N6M0095KX | N7G0790KX | N7L0132KX | N8D0483KX |
| | | N7M0592KX | N5M0693KX | N6M0113KX | N7H0012KX | N7L0856KX | N8D0594KX |
| | | N8A0203KX | N5M0662KX | N6M0263KX | N7H0347KX | N7L1004KX | N8D0647KX |
| | | N4D0338KX | N5M0732KX | N7A0293KX | N7G0904KX | N7L0475KX | N8D0311KX |
| | | N4D0328KX | N5M0741KX | N7A0792KX | N7H0174KX | N7K1181KX | N8D0536KX |
| | | N4E0846KX | N6A0913KX | N7B0125KX | N7H0470KX | N7K1183KX | N8D0694KX |
| | | N4E0796KX | N6B0332KX | N7B0154KX | N7H0569KX | N7L0811KX | N8D0482KX |
| | | N4E1018KX | N6C0462KX | N7B0177KX | N7H0520KX | N7M0100KX | N8E0435KX |
| | | N4E1059KX | N6D0388KX | N7B0070KX | N7H0315KX | N7M0132KX | N8E0431KX |
| | | N4E0980KX | N6E0078KX | N7B0946KX | N7H0419KX | N7M0506KX | N8E0498KX |
| | | N4E1654KX | N6E0099KX | N7B0905KX | N7H1017KX | N7M0554KX | N8E0659KX |
| | | N4E1173KX | N6E0129KX | N7B0885KX | N7H1057KX | N7M0778KX | N8E0725KX |
| | | N4E1655KX | N6E0366KX | N7C0387KX | N7H1096KX | N7M0589KX | N8E0567KX |
| | | N4E1269KX | N6E0416KX | N7C0442KX | N7H1141KX | N7M0620KX | N8D1045KX |
| | | N4G0523KX | N6D0506KX | N7C0517KX | N7H1187KX | N7M0918KX | N8E0612KX |
| | | N4H0078KX | N6D0474KX | N7C0405KX | N7J0070KX | N7M0614KX | N8E0700KX |
| | | N4G1541KX | N6F0039KX | N7C0775KX | N7J0099KX | N7M1000KX | N8E0149KX |
| | | N4J0215KX | N6F0088KX | N7C0812KX | N7H0957KX | N7M0904KX | N8E0201KX |
| | | N4J0250KX | N6F0156KX | N7C0899KX | N7J0135KX | N7M0967KX | N8D1067KX |
| | | N4J0271KX | N6F0720KX | N7C0857KX | N7J0174KX | N8A0380KX | N8F0094KX |
| | | N4J1002KX | N6F0780KX | N7C0927KX | N7H0712KX | N8A0381KX | N8F0307KX |
| | | N4J1247KX | N6F0816KX | N7C0977KX | N7H1175KX | N8A0772KX | N8F0183KX |
| | | N4J1217KX | N6G0243KX | N7D0135KX | N7J0483KX | N8A0960KX | N8F0258KX |
| | | N4K1465KX | N6G0280KX | N7D0247KX | N7H0786KX | N8A1028KX | N8F0335KX |
| | | N4M0921KX | N6G0198KX | N7C0485KX | N7J0550KX | N8A1050KX | N8F0347KX |
| | | N5B0340KX | N6G0139KX | N7D0101KX | N7J0508KX | N8A0065KX | N8F0383KX |
| | | N5B0417KX | N6G0191KX | N7C1014KX | N7J0692KX | N8A1119KX | N8F0482KX |
| | | N5B0484KX | N6G0213KX | N7D0182KX | N7J0858KX | N8B0204KX | N8B1048KX |
| | | N5B0285KX | N6G0099KX | N7D0226KX | N7J0636KX | N8A1099KX | N8F0428KX |
| | | N5B0680KX | N6H0168KX | N7D0264KX | N7J0897KX | N8B0114KX | N8D0394KX |
| | | N5D0467KX | N6H0291KX | N7D0495KX | N7J1014KX | N8B0137KX | N8E0233KX |
| | | N5D0497KX | N6H0327KX | N7D0416KX | N7K0270KX | N8B0571KX | N8E0266KX |
| | | N5D0550KX | N6H0370KX | N7D0538KX | N7K0307KX | N8B0666KX | N8F0691KX |
| | | N5D0609KX | N6H0433KX | N7D0591KX | N7J1155KX | N8B0805KX | N8F0773KX |
| | | N5D0660KX | N6H0466KX | N7D0918KX | N7K0225KX | N8B0885KX | N8E0378KX |
| | | N5D0736KX | N6E0117KX | N7D0473KX | N7K0500KX | N8B0641KX | N8F0658KX |
| | | N5D0765KX | N6H0247KX | N7D0945KX | N7K0008KX | N8B0709KX | N8F0858KX |
| | | N5D1008KX | N6H0456KX | N7E0372KX | N7K0085KX | N8C0159KX | N8F0879KX |
| | | N5E0068KX | N6H0827KX | N7E0518KX | N7K0154KX | N8B0849KX | N8F1015KX |
| | | N5E0143KX | N6H0792KX | N7E0565KX | N7K0272KX | N8B1009KX | N8F0976KX |
| | | N5E0297KX | N6J0152KX | N7E0610KX | N7K0303KX | N8B1034KX | N8E0321KX |
| | | N5E0727KX | N6J0185KX | N7E0920KX | N7K0400KX | N8B0769KX | N8E0699KX |
| | | N5E0756KX | N6J0243KX | N7E0962KX | N7K0443KX | N8B0958KX | N8F1046KX |
| | | N5F0096KX | N6J0476KX | N7F0093KX | N7K0544KX | N8C0324KX | N8E0081KX |
| | | N5F1223KX | N6K0209KX | N7F0186KX | N7K0956KX | N8C0340KX | N8G0871KX |
| | | N5F1382KX | N6K0676KX | N7F0094KX | N7K0562KX | N8C0470KX | N8G0942KX |
| | | N5G0414KX | N6K0739KX | N7F0663KX | N7K0711KX | N8C0546KX | N8F0921KX |
| | | N5G0329KX | N6K0774KX | N7F0696KX | N7K0742KX | N8C0718KX | N8G0911KX |
| | | N5G0355KX | N6K0824KX | N7F0745KX | N7K0783KX | N8B0905KX | N8H0735KX |
| | | N5G0868KX | N6K0251KX | N7F0224KX | N7K0087KX | N8C0589KX | N8H0759KX |
| N5G0244KX | N6K0871KX | N7F1052KX | N7K0821KX | N8C0779KX | N8H0683KX | | |
| N5G0339KX | N6K0293KX | N7F1139KX | N7L0057KX | N8C0777KX | N8G0980KX | | |
| N5G0405KX | N6K1137KX | N7G0016KX | N7L0091KX | N8C0816KX | N8J0819KX | | |
| N5G0809KX | N6K0160KX | N7F0662KX | N7L0403KX | N8C1338KX | N8L0895KY | | |
| N5J1153KX | N6L0909KX | N7G0241KX | N7K1052KX | N8D0083KX | N9B0348KY | | |
| N5J1201KX | N6L0942KX | N7G0279KX | N7L0431KX | N8C1339KX | N6L0814KRX | | |
| N5K0045KX | N6M0065KX | N7F1019KX | N7L0592KX | N8D0043KX | N8M0168KRY | | |
| N5K0089KX | N6M0134KX | N7F0822KX | N7K1150KX | N8D0142KX | N9A0665KRY | | |
| N5K0115KX | N6L0806KX | N7G0498KX | N7L0361KX | N8D0312KX | N9A0954KRY | | |
| N5J1233KX | N6L0944KX | | | | | | |



Attachment A: Affected Product List

| Item | Description | Affected Lot Numbers | | | | | |
|-----------|---|----------------------|-----------|-----------|-----------|-----------|-----------|
| EGIA60AMT | Endo GIA™ 60 mm Articulating Medium/Thick Reload with Tri-Staple™ Technology | N4C1545KX | N5J0532KX | N6D0903KX | N7F0868KX | N8G0752KX | N8L0327KY |
| | | N4D0042KX | N5J0658KX | N6D0924KX | N7F0923KX | N8G0839KX | N8L0440KY |
| | | N4C1718KX | N5J0816KX | N6E0002KX | N7F0589KX | N8G0879KX | N8L0163KY |
| | | N4D0366KX | N5H0100KX | N6E0244KX | N7F0903KX | N8G0880KX | N8L0204KY |
| | | N4D0453KX | N5H0481KX | N6E0287KX | N7F0964KX | N8G1025KX | N8L0351KY |
| | | N4D0589KX | N5J0418KX | N6E0335KX | N7G0062KX | N8H0016KX | N8K1023KY |
| | | N4D0605KX | N5J0572KX | N6D0607KX | N7G0209KX | N8H0089KX | N8L0099KY |
| | | N4D0688KX | N5J0694KX | N6D0285KX | N7F0961KX | N8H0135KX | N8L0368KY |
| | | N4D0814KX | N5J0966KX | N6D0536KX | N7G0246KX | N8G0217KX | N8L0471KY |
| | | N4D0897KX | N5J0775KX | N6D0580KX | N7G0287KX | N8H0372KX | N8L0594KY |
| | | N4D1145KX | N5J1099KX | N6D0160KX | N7G0689KX | N8G0936KX | N8L0534KY |
| | | N4E1383KX | N5J1130KX | N6D0180KX | N7G0723KX | N8H0159KX | N8L0765KY |
| | | N4E1505KX | N5H0970KX | N6D0230KX | N7G0775KX | N8F1045KX | N8L0527KY |
| | | N4F0006KX | N5K0139KX | N6D0283KX | N7G0661KX | N8G0983KX | N8L0570KY |
| | | N4F0076KX | N5K0208KX | N6D0334KX | N7H0396KX | N8H0175KX | N8L0733KY |
| | | N4F0114KX | N5K0308KX | N6D0376KX | N7H0493KX | N8H0220KX | N8L0633KY |
| | | N4F0516KX | N5K0374KX | N6D0672KX | N7H0102KX | N8H0298KX | N8M0004KY |
| | | N4F0853KX | N5K0412KX | N6E0068KX | N7H0762KX | N8H0332KX | N8L0715KY |
| | | N4F0719KX | N5K0429KX | N6E0573KX | N7H0827KX | N8H0406KX | N8L0772KY |
| | | N4F0929KX | N5K0564KX | N6E0600KX | N7H0422KX | N8H0506KX | N8L0822KY |
| | | N4F0695KX | N5K0629KX | N6E0691KX | N7H0464KX | N8H0600KX | N8M0026KY |
| | | N4F0981KX | N5K0665KX | N6E0652KX | N7H0716KX | N8H0613KX | N8M0035KY |
| | | N4F1092KX | N5K0700KX | N6F0165KX | N7H0867KX | N8H0868KX | N8L0073KY |
| | | N4F1331KX | N5K0732KX | N6F0232KX | N7J0254KX | N8H0434KX | N8M0064KY |
| | | N4F1450KX | N5K0848KX | N6F0279KX | N7J0371KX | N8H0474KX | N8M0100KY |
| | | N4F1163KX | N5K0895KX | N6B0081KX | N7J0420KX | N8H0776KX | N8M0178KY |
| | | N4F1542KX | N5K0941KX | N6E0462KX | N7J0454KX | N8H0574KX | N8M0140KY |
| | | N4F1159KX | N5K0775KX | N6F0307KX | N7J0214KX | N8H0606KX | N8M0243KY |
| | | N4F1383KX | N5K0992KX | N6F0385KX | N7J0286KX | N8H0652KX | N8L0673KY |
| | | N4H0432KX | N5K1054KX | N6F0087KX | N7J0845KX | N8H0750KX | N8M0217KY |
| | | N4H0475KX | N5K0595KX | N6F0442KX | N7J0386KX | N8H0800KX | N8M0262KY |
| | | N4H0503KX | N5L0595KX | N6F0397KX | N7J0748KX | N8H0969KX | N8M0276KY |
| | | N4J1170KX | N5M0150KX | N6F0474KX | N7J0792KX | N8H0653KX | N8M0351KY |
| | | N4K0742KX | N5L0809KX | N6F0512KX | N7J0884KX | N8J0221KX | N8M0439KY |
| | | N4K0835KX | N5M0088KX | N6F0541KX | N7K1119KX | N8J0030KX | N8M0459KY |
| | | N4K0870KX | N5L0746KX | N6F0573KX | N7K1094KX | N8J0260KX | N8M0551KY |
| | | N4K0212KX | N5M0536KX | N6C0753KX | N7L0635KX | N8H0815KX | N8M0628KY |
| | | N4K0757KX | N5M0491KX | N6F0211KX | N7L0737KX | N8J0494KX | N8M0673KY |
| | | N4L0249KX | N5M0582KX | N6F0611KX | N7L0738KX | N8J0188KX | N8M0512KY |
| | | N4L0315KX | N5M0769KX | N6F0656KX | N7M0072KX | N8J0319KX | N8M0699KY |
| | | N4M1062KX | N5M0827KX | N6F0659KX | N7L0894KX | N8J0443KX | N8M0621KY |
| | | N4M0982KX | N5M0459KX | N6F0705KX | N7M0322KX | N8J0301KX | N8M0394KY |
| N5A0364KX | N5M0632KX | N6F0935KX | N7L0957KX | N8J0414KX | N8M0588KY | | |
| N5A0345KX | N6A0004KX | N6J0202KX | N7M0323KX | N8J0564KX | N8M0713KY | | |
| N5A0286KX | N6A0034KX | N6J0314KX | N7M0864KX | N8J0617KX | N8M0793KY | | |
| N5A1461KX | N6A0086KX | N6J0364KX | N7M0785KX | N8J0686KX | N8M0796KY | | |
| N5B0700KX | N6A0124KX | N6K0742KX | N8A0098KX | N8J0762KX | N8M0763KY | | |
| N5B0757KX | N6A0520KX | N6L0523KX | N8A0744KX | N8H0894KX | N9A0013KY | | |
| N5B0031KX | N6A0527KX | N6L0634KX | N8A0882KX | N8J0060KX | N9A0092KY | | |
| N5B0794KX | N6A0428KX | N6L0946KX | N8A0818KX | N8J0313KX | N9A0171KY | | |
| N5B0856KX | N6B0052KX | N6L0527KX | N8A0911KX | N8J0523KX | N8M0849KY | | |

Attachment A: Affected Product List

| Item | Description | Affected Lot Numbers | | | | | |
|-----------|---|----------------------|-----------|-----------|-----------|-----------|------------|
| EGIA60AMT | Endo GIA™ 60 mm Articulating Medium/Thick Reload with Tri-Staple™ Technology | N5C0465KX | N6B0028KX | N6L0556KX | N8A1044KX | N8J0534KX | N9A0139KY |
| | | N5C0639KX | N6B0135KX | N6M0737KX | N8B0372KX | N8J0810KX | N9A0329KY |
| | | N5C0725KX | N6B0105KX | N6M0329KX | N8B0411KX | N8J0102KX | N9A0151KY |
| | | N5C0986KX | N6B0177KX | N6M0556KX | N8B0458KX | N8J0882KX | N8M0711KY |
| | | N5C0668KX | N6B0270KX | N6M0698KX | N8B0311KX | N8J0952KX | N9A0358KY |
| | | N5C0914KX | N6B0219KX | N7A0131KX | N8A0819KX | N8J0787KX | N9A0460KY |
| | | N5E0348KX | N6B0323KX | N7A0167KX | N8B0497KX | N8J1064KX | N9A0497KY |
| | | N5D0862KX | N6B0386KX | N7A0208KX | N8C0112KX | N8J1019KX | N9A0423KY |
| | | N5F0181KX | N6B0477KX | N6M0729KX | N8B0553KX | N8J0723KX | N8M0820KY |
| | | N5F0330KX | N6B0405KX | N7A0227KX | N8C0638KX | N8J0852KX | N9A0366KY |
| | | N5E1158KX | N6B0504KX | N7A0086KX | N8D0183KX | N8H0940KX | N9A0400KY |
| | | N5E1245KX | N6B0541KX | N7A0941KX | N8D0226KX | N8J0005KX | N9A0186KY |
| | | N5F0260KX | N6B0624KX | N7B0024KX | N8D0242KX | N8J0990KX | N9A0267KY |
| | | N5F0348KX | N6B0722KX | N7B0123KX | N8D0314KX | N8J1070KX | N9A0544KY |
| | | N5C0758KRX | N6B0746KX | N7B0345KX | N8E1013KX | N8K0016KX | N9A0636KY |
| | | N5E1213KX | N6B0788KX | N7B0591KX | N8E1175KX | N8K0081KX | N9A0652KY |
| | | N5G0114KX | N6B0852KX | N7B0739KX | N8E1177KX | N8K0110KX | N9A0765KY |
| | | N5G0154KX | N6B0923KX | N7C0332KX | N8E0676KX | N8K0180KX | N9A0725KY |
| | | N5G0186KX | N6A0253KX | N7C0275KX | N8E0687KX | N8K0212KX | N9A0787KY |
| | | N5G0877KX | N6A0255KX | N7C0371KX | N8E1135KX | N8K0231KX | N8L0853KY |
| | | N5G0983KX | N6C0398KX | N7C0833KX | N8F0098KX | N8K0319KX | N9A0618KY |
| | | N5G1011KX | N6C0355KX | N7C0959KX | N8E0814KX | N8K0136KX | N9A0812KY |
| | | N5G1122KX | N6C0369KX | N7C0756KX | N8F0581KX | N8K0361KX | N9A0827KY |
| | | N5G0660KX | N6C0370KX | N7C0799KX | N8F0523KX | N8K0452KX | N9A0880KY |
| | | N5G1147KX | N6C0524KX | N7D0453KX | N8G0035KX | N8K0393KX | N9A0937KY |
| | | N5G1248KX | N6C0589KX | N7E0043KX | N8G0067KX | N8K0492KX | N9A0521KY |
| | | N5H0250KX | N6C0562KX | N7E0148KX | N8E1060KX | N8K0254KX | N9A0696KY |
| | | N5H0623KX | N6C0509KX | N7D0592KX | N8G0179KX | N8K0584KX | N9B0020KY |
| | | N5H0676KX | N6C0598KX | N7D0632KX | N8F1111KX | N8K0762KY | N9B0049KY |
| | | N5H0226KX | N6C0678KX | N7D0690KX | N8G0244KX | N8K0632KX | N9B0220KY |
| | | N5H0730KX | N6C0701KX | N7E0223KX | N8G0122KX | N8K0421KX | N9B0122KY |
| | | N5H0838KX | N6C0737KX | N7D0908KX | N8G0284KX | N8K0697KY | N9B0084KY |
| | | N5H0949KX | N6C0800KX | N7E0304KX | N8F0619KX | N8K0709KY | N9B0299KY |
| | | N5H1013KX | N6C0841KX | N7D0744KX | N8G0566KX | N8K0898KY | N9B0522KY |
| | | N5H1082KX | N6C0889KX | N7D0949KX | N8G0624KX | N8K0939KY | N9B0066KY |
| | | N5H1111KX | N6C0913KX | N7E0109KX | N8E0633KX | N8K0842KY | N9B0298KY |
| | | N5H1158KX | N6C0938KX | N7E0188KX | N8G0202KX | N8K0437KX | N9B0201KY |
| | | N5H1186KX | N6C0972KX | N7E0726KX | N8G0388KX | N8K0800KY | N9B0389KY |
| | | N5H0993KX | N6D0009KX | N7E0806KX | N8G0440KX | N8K0971KY | N9B0570KY |
| | | N5J0220KX | N6D0051KX | N7E0605KX | N8G0351KX | N8K0996KY | N9B0490KY |
| | | N5J0266KX | N6D0070KX | N7F0157KX | N8G0468KX | N8K1063KY | N9B0540KY |
| | | N5J0295KX | N6D0091KX | N7F0307KX | N8G0489KX | N8L0016KY | N9B0722KY |
| | | N5J0321KX | N6D0715KX | N7F0173KX | N8G0513KX | N8L0072KY | N9B0403KY |
| | | N5H0762KX | N6D0735KX | N7E0990KX | N8G0688KX | N8K0531KX | N9C0450KY |
| | | N5J0492KX | N6D0757KX | N7F0370KX | N8G0780KX | N8K0983KY | N5L0673KRX |
| | | N5H1216KX | N6D0796KX | N7F0500KX | N8G0550KX | N8L0234KY | N9A0773KRX |
| | | N5J0512KX | N6D0830KX | N7F0061KX | N8G0697KX | N8L0283KY | |

Identifying Affected Product

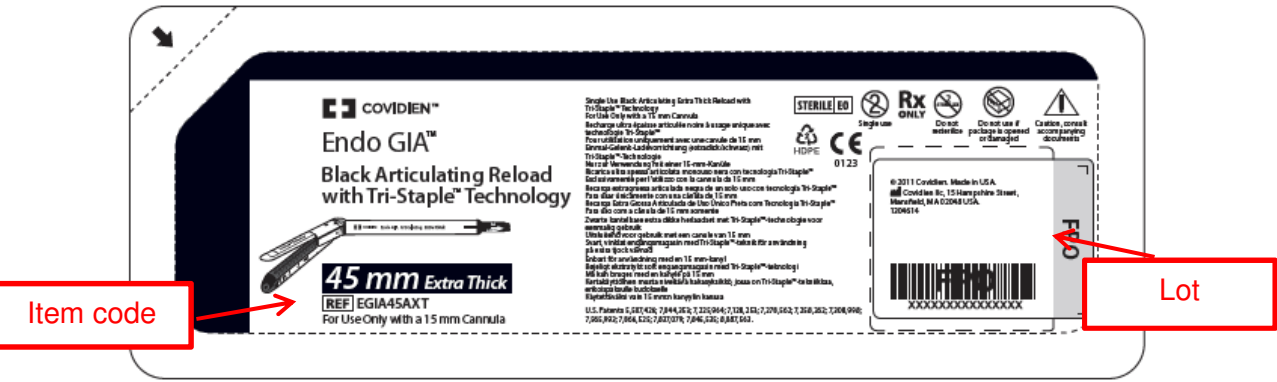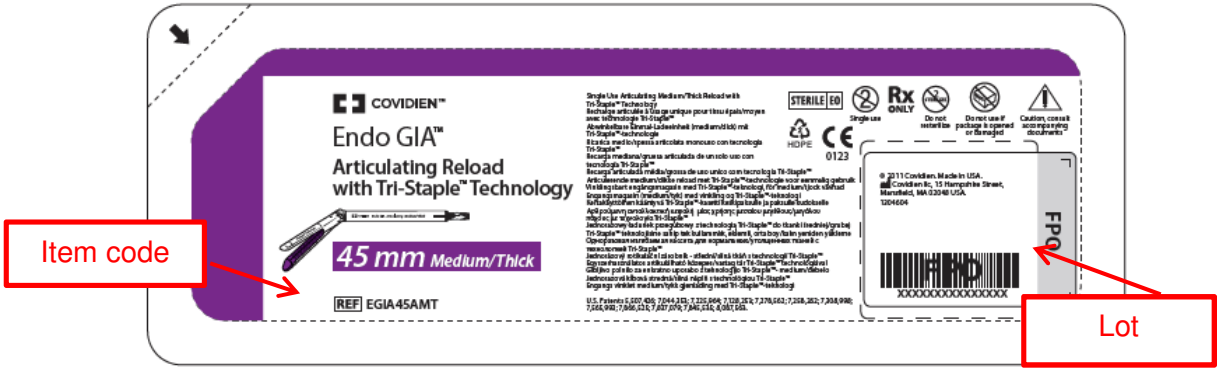

Parent Code

ORDER NO: 19936878 67090146 KIT PARENT: 00Z2199

NAME: MARY HITCHCOCK MEMORIAL HOSP

| Qty | Code | Description | Qty | Code | Description |
|-----|-----------|----------------------------|-----|------|-------------|
| 1 | EGIARADIT | EGIA RADIAL MED THK RELOAD | | | |
| 1 | EGIAUSTND | EGIA 4 HANDLE STANDARD | | | |

