

URGENT FIELD SAFETY NOTICE

RA2020 - 2310673

LIFEPAK® 500 Automated External Defibrillator (AED)

FSCA Identifier: RA2020-2310673

Type of Action: Field Safety Corrective Action

Products Affected: LIFEPAK 500 AEDs

Catalog Numbers: 97401-000001 to 97401-000015
99401-000001 to 99401-000388

Part Numbers / Serial Numbers Please see the attached list (Appendix I).
All serial numbers.

23rd June 2020

Dear Valued Customer,

Stryker is conducting a Voluntary Correction for all **LIFEPAK 500 AEDs** that may still be in use and, as a result, may experience a component wear out issue that can prevent the device from detecting a patient connection. We previously announced the discontinuation of repair, parts, and support on multiple occasions. Our most recent notification, in February 2020, noted that LIFEPAK 500 devices will no longer be supported, the LIFEPAK 500 AED batteries and other accessories were discontinued as of February 3, 2020, and electrodes will be discontinued February 3, 2021.

Description of issue

Stryker has become aware that LIFEPAK 500 AEDs in high-use environments (Emergency Medical Services) may not detect a patient connection due to mechanical wear-through of the contact plating on the therapy connector. Wear-through of the connector exposes base metal on which an oxide layer may form and result in the device not recognizing a patient is connected. When this circumstance arises, the device will provide the user the "CONNECT ELECTRODES" message.

This mechanical wear-through is the result of a high volume of insertion/removal cycles for the therapy electrodes and has only been observed at a single customer with a high-use environment. It has also been observed that removing and reinstalling the electrodes or replacing the electrodes may reestablish the patient connection and allow treatment to continue. There have been five Adverse Event Reports where the device failed to initially recognize a patient connection which resulted in a delay to treatment.

Identification of Impacted Product

All LIFEPAK 500 AEDs that remain in use are impacted.

Stryker's Planned Actions

The Company is notifying all customers with LIFEPAK 500 devices, regardless of age and use frequency, to make them aware of this potential safety issue, the need for device replacement and to provide additional warnings and cautions to include as supplemental labeling for any devices that are within their expected life.

Required customer actions

We request that you read this notice carefully and complete the following actions:

1. **If you experience a "CONNECT ELECTRODES" voice prompt with the LIFEPAK 500, immediately remove and reinstall the electrodes to the device or replace the electrodes with your spare electrodes and check patient connection. If "CONNECT ELECTRODES" voice prompt continues, immediately obtain a backup device and remove the LIFEPAK 500 from use.**
2. Continue to perform the External Test Load Test per Maintenance section of the LIFEPAK 500 Operating Instructions and your local protocols as this test may identify this issue outside of patient care. If during this test, a "CONNECT ELECTRODES" voice prompt message is received, immediately remove the device from service.
3. For any devices that are still within their expected life (meaning it has not been more than eight years since the original purchase of the new device from Physio-Control/Stryker or its authorized distributors), include the attached additional warnings and cautions as supplemental labeling for your device.
4. Review your attached affected device list and confirm if your device is still in service, transferred to a new location or is no longer in service. Please return the confirmation sheet to **raqa.uk@stryker.com**

We strongly recommend the replacement of any LIFEPAK 500 device as all support for the product has been discontinued. The previously referenced February 2020 notice regarding discontinuation of the LIFEPAK 500 device, support and accessories can be accessed at: <https://www.strykeremergencycare.com/service--support-overview/end-of-life-notice/>.

As stated above, we understand the effects this situation may have on your organization and would like to assist you in replacing your LIFEPAK 500 device with the FDA-approved LIFEPAK CR2 AED, LIFEPAK 1000 defibrillator or the HeartSine defibrillators based on your needs. Contact your local Stryker sales representative or authorized distributor to discuss trade-up or flexible financing options.

We request that you respond to this notice within **7 calendar days** from the date of receipt.

The LIFEPAK 500AED has an eight (8) year expected life. Using the device beyond 8 years may result in excessive wear and damage of components within the Therapy Connector. Devices should not be in use beyond the eight (8) year expected life.

Your designated contact person for this action is given below. Should you have any queries concerning this matter please do not hesitate to contact them directly.

Name: Sharun Thavarajan
Position: Senior RAQA Specialist
E-mail: sharunyan.thavarajan@stryker.com
Phone: +44(0)7929 021 221

In line with the recommendations of the Meddev Vigilance Guidance document Ref 2.12-1, we can confirm that this FSCA has been notified appropriately to the National Competent Authority for your country.

On behalf of Stryker we thank you sincerely for your help and support in completing this action within the target date and regret any inconvenience that may be caused. We would like to reassure you that Stryker is committed to ensuring that only conforming devices, meeting our high internal quality standards, remain on the market.

Yours faithfully,

Sharun Thavarajan
Regulatory Affairs and Quality Assurance

**URGENT FIELD SAFETY NOTICE: RA2020 – 2310673
BUSINESS REPLY FORM**

FSCA Identifier: RA2020-2310673
Type of Action: Field Safety Corrective Action
Products Affected: LIFEPAK 500 AEDs
Product Catalog Numbers: 97401-000001 to 97401-000015
 99401-000001 to 99401-000388
 Please see the attached list (Appendix I).

I acknowledge receipt of the Field Safety Notice for RA2020-2310673 and can confirm that:

We have not located any of these devices in our inventory: (please delete if not applicable)			
We have located the following devices (attach additional sheet if needed):			
Catalog Number	Part Number	Serial Number	In service or Not in service or Transferred?
We have further distributed subject devices to the following organizations. I also agree to further distribute and communicate this important information from this letter to those whom I have distributed any of the devices noted in this letter.			
Facility Name			
Facility Address			

Please sign and return this form to acknowledge receipt of the product notice.			
Name of Hospital / Organisation		Department	
Contact Name		Address	
Contact Title			
Contact Signature		E-mail Address	
Contact Phone No.		Date	

PLEASE COMPLETE THIS FORM WITHIN 7 CALENDAR DAYS AND RETURN IT BY USING THE EMAIL: raqa.uk@stryker.com

RA2020 - 2310673

Appendix I - Affected Catalog and Part numbers.

All LIFEPAK 500 AEDs that remain in use are impacted.

Catalog Number	Part Number
97401-000001	U3005400-000
97401-000002	u3005400-000002
97401-000002	U3005400-000014
97401-000002	U3005400-000059
97401-000003	U3005400-000003
97401-000004	U3005400-000004
97401-000004	U3005400-000056
97401-000005	U3005400-000005
97401-000005	U3005400-000058
97401-000006	U3005400-000057
97401-000007	U3005400-009
97401-000008	U3005400-020
97401-000009	u3005400-029
97401-000010	U3005400-064
97401-000011	U3005400-070
97401-000012	U3005400-200
97401-000013	U3005400-220
97401-000014	U3005400-300
97401-000015	U3005400-320
97401-000016	U3005400-332
97401-000017	U3011790-000001
97401-000018	u3011790-000009
97401-000019	U3011790-000011
97401-000020	U3011790-000012
97401-000021	U3011790-000097
97401-000021	U3011790-000121
97401-000022	U3011790-000099
97401-000022	U3011790-000127
97401-000023	U3011790-000100
97401-000023	u3011790-000113
97401-000024	U3011790-000102
97401-000024	u3011790-000114
97401-000028	U3011790-001129
97401-000030	U3011790-001131
97401-000031	u3011790-001133
97401-000032	U3011790-001132
97401-000034	U3011790-001343

Catalog Number	Part Number
97401-000039	U3011790-001361
97401-000040	U3011790-001362
99401-000001	3005400-000
99401-000037	3005400-001
99401-000038	3005400-002
99401-000039	3005400-003
99401-000040	3005400-004
99401-000041	3005400-005
99401-000042	3005400-009
99401-000117	3005400-010
99401-000118	3005400-011
99401-000120	3005400-013
99401-000121	3005400-014
99401-000123	3005400-018
99401-000124	3005400-020
99401-000125	3005400-021
99401-000126	3005400-022
99401-000127	3005400-029
99401-000128	3005400-030
99401-000129	3005400-031
99401-000130	3005400-064
99401-000131	3005400-065
99401-000132	3005400-066
99401-000133	3005400-067
99401-000134	3005400-068
99401-000135	3005400-070
99401-000136	3005400-071
99401-000139	3005400-074
99401-000140	3005400-076
99401-000141	3005400-078
99401-000142	3005400-082
99401-000143	3005400-083
99401-000144	3005400-084
99401-000145	3005400-085
99401-000146	3005400-200
99401-000148	3005400-203
99401-000149	3005400-212

Catalog Number	Part Number
99401-000150	3005400-220
99401-000151	3005400-225
99401-000152	3005400-229
99401-000153	3005400-300
99401-000154	3005400-301
99401-000155	3005400-302
99401-000156	3005400-303
99401-000157	3005400-304
99401-000158	3005400-305
99401-000159	3005400-307
99401-000161	3005400-310
99401-000162	3005400-311
99401-000163	3005400-315
99401-000164	3005400-320
99401-000165	3005400-321
99401-000166	3005400-322
99401-000167	3005400-324
99401-000168	3005400-329
99401-000169	3005400-332
99401-000170	3005400-333
99401-000200	3011790-000999
99401-000201	3005400-000001
99401-000201	3005400-000186
99401-000202	3005400-000144
99401-000203	3005400-000002
99401-000203	3005400-000059
99401-000204	3005400-000014
99401-000205	3005400-000060
99401-000206	3005400-000003
99401-000206	3005400-000057
99401-000207	3005400-000020
99401-000208	3005400-000045
99401-000208	3005400-000071
99401-000209	3005400-000079
99401-000210	3005400-000004
99401-000210	3005400-000056
99401-000211	3005400-000005

Catalog Number	Part Number
99401-000211	3005400-000058
99401-000212	3005400-000088
99401-000213	3005400-000091
99401-000214	3005400-000006
99401-000214	3005400-000073
99401-000215	3005400-000035
99401-000216	3005400-000007
99401-000217	3005400-000012
99401-000218	3005400-000013
99401-000219	3005400-000084
99401-000220	3005400-000008
99401-000220	3005400-000123
99401-000221	3005400-000017
99401-000222	3005400-000042
99401-000222	3005400-000111
99401-000223	3005400-000100
99401-000224	3005400-000148
99401-000225	3005400-000009
99401-000225	3005400-000076
99401-000226	3005400-000010
99401-000227	3005400-000011
99401-000227	3005400-000085
99401-000228	3005400-000021
99401-000228	3005400-000062
99401-000229	3005400-000117
99401-000230	3005400-000125
99401-000231	3005400-000016
99401-000231	3005400-000112
99401-000232	3005400-000087
99401-000233	3005400-000121
99401-000234	3005400-000018
99401-000234	3005400-000128
99401-000235	3005400-000030
99401-000235	3005400-000102
99401-000236	3005400-000118
99401-000237	3005400-000019
99401-000238	3005400-000022
99401-000238	3005400-000074
99401-000239	3005400-000029
99401-000239	3005400-000068
99401-000240	3005400-000106
99401-000241	3005400-000023

Catalog Number	Part Number
99401-000242	3005400-000034
99401-000242	3005400-000116
99401-000243	3005400-000024
99401-000244	3005400-000040
99401-000244	3005400-000114
99401-000245	3005400-000025
99401-000246	3005400-000027
99401-000247	3005400-000120
99401-000248	3005400-000142
99401-000249	3005400-000028
99401-000249	3005400-000070
99401-000250	3005400-000032
99401-000250	3005400-000077
99401-000251	3005400-000043
99401-000251	3005400-000090
99401-000252	3005400-000048
99401-000253	3005400-000089
99401-000254	3005400-000132
99401-000255	3005400-000139
99401-000256	3005400-000147
99401-000257	3005400-000033
99401-000257	3005400-000096
99401-000258	3005400-000036
99401-000258	3005400-000105
99401-000259	3005400-000041
99401-000259	3005400-000078
99401-000260	3005400-000047
99401-000260	3005400-000064
99401-000261	3005400-000072
99401-000262	3005400-000081
99401-000263	3005400-000082
99401-000264	3005400-000103
99401-000265	3005400-000130
99401-000266	3005400-000138
99401-000267	3005400-000143
99401-000268	3005400-000145
99401-000269	3005400-000146
99401-000270	3005400-000149
99401-000271	3005400-000037
99401-000272	3005400-000038
99401-000273	3005400-000039
99401-000274	3005400-000119

Catalog Number	Part Number
99401-000275	3005400-000044
99401-000275	3005400-000135
99401-000276	3005400-000050
99401-000276	3005400-000092
99401-000277	3005400-000051
99401-000277	3005400-000153
99401-000278	3005400-000080
99401-000279	3005400-000127
99401-000280	3005400-000140
99401-000281	3005400-000141
99401-000282	3005400-000046
99401-000282	3005400-000097
99401-000283	3005400-000049
99401-000283	3005400-000086
99401-000284	3005400-000104
99401-000285	3005400-000136
99401-000286	3005400-000052
99401-000286	3005400-000156
99401-000287	3005400-000099
99401-000288	3005400-000124
99401-000289	3005400-000129
99401-000290	3005400-000131
99401-000291	3005400-000134
99401-000292	3005400-000053
99401-000293	3005400-000054
99401-000293	3005400-000083
99401-000294	3005400-000063
99401-000295	3005400-000069
99401-000296	3005400-000126
99401-000297	3005400-000075
99401-000298	3005400-000095
99401-000299	3005400-000098
99401-000300	3005400-000137
99401-000301	3005400-000101
99401-000302	3005400-000113
99401-000303	3005400-000115
99401-000304	3011790-000001
99401-000304	3011790-000037
99401-000305	3011790-000047
99401-000306	3011790-000048
99401-000307	3011790-000049
99401-000308	3011790-000060

Catalog Number	Part Number
99401-000309	3011790-000009
99401-000310	3011790-000038
99401-000311	3011790-000045
99401-000312	3011790-000046
99401-000313	3011790-000065
99401-000314	3011790-000067
99401-000315	3011790-000093
99401-000316	3011790-000103
99401-000318	3011790-000011
99401-000319	3011790-000012
99401-000320	3011790-000019
99401-000321	3011790-000034
99401-000322	3011790-000044
99401-000323	3011790-000091
99401-000324	3011790-000013
99401-000325	3011790-000073
99401-000326	3011790-000074
99401-000327	3011790-000076
99401-000328	3011790-000088
99401-000329	3011790-000014
99401-000330	3011790-000018
99401-000331	3011790-000031
99401-000332	3011790-000015
99401-000333	3011790-000016
99401-000334	3011790-000030
99401-000335	3011790-000075
99401-000336	3011790-000020
99401-000337	3011790-000021
99401-000338	3011790-000058
99401-000339	3011790-000022
99401-000340	3011790-000057
99401-000341	3011790-000059
99401-000342	3011790-000062
99401-000343	3011790-000063
99401-000344	3011790-000023
99401-000345	3011790-000082
99401-000346	3011790-000024
99401-000347	3011790-000094
99401-000348	3011790-000025
99401-000349	3011790-000081
99401-000350	3011790-000095
99401-000351	3011790-000026

Catalog Number	Part Number
99401-000352	3011790-000066
99401-000353	3011790-000083
99401-000354	3011790-000027
99401-000355	3011790-000056
99401-000356	3011790-000090
99401-000357	3011790-000028
99401-000358	3011790-000033
99401-000359	3011790-000084
99401-000360	3011790-000029
99401-000361	3011790-000036
99401-000362	3011790-000087
99401-000363	3011790-000107
99401-000364	3011790-000039
99401-000365	3011790-000104
99401-000366	3011790-000040
99401-000367	3011790-000089
99401-000368	3011790-000098
99401-000369	3011790-000041
99401-000370	3011790-000042
99401-000371	3011790-000053
99401-000372	3011790-000092
99401-000373	3011790-000101
99401-000374	3011790-000043
99401-000375	3011790-000052
99401-000376	3011790-000061
99401-000377	3011790-000064
99401-000378	3011790-000096
99401-000379	3011790-000068
99401-000380	3011790-000069
99401-000381	3011790-000070
99401-000382	3011790-000072
99401-000383	3011790-000079
99401-000384	3011790-000080
99401-000385	3011790-000085
99401-000386	3011790-000086
99401-000387	3011790-000097
99401-000387	3011790-000121
99401-000388	3011790-000099
99401-000388	3011790-000127
99401-000389	3011790-000172
99401-000390	3011790-000100
99401-000390	3011790-000113

Catalog Number	Part Number
99401-000391	3011790-000142
99401-000392	3011790-000102
99401-000392	3011790-000114
99401-000393	3011790-000160
99401-000394	3011790-001005
99401-000395	3011790-000106
99401-000395	3011790-000146
99401-000396	3011790-000122
99401-000397	3011790-000151
99401-000398	3011790-000123
99401-000399	3011790-000124
99401-000400	3011790-000179
99401-000401	3011790-000195
99401-000402	3011790-001021
99401-000403	3011790-000125
99401-000404	3011790-000181
99401-000405	3011790-000126
99401-000406	3011790-000157
99401-000407	3011790-000197
99401-000408	3011790-000129
99401-000409	3011790-000132
99401-000410	3011790-000135
99401-000411	3011790-000137
99401-000412	3011790-000145
99401-000413	3011790-000155
99401-000414	3011790-000165
99401-000415	3011790-001019
99401-000416	3011790-001020
99401-000417	3011790-001044
99401-000418	3011790-000131
99401-000419	3011790-000169
99401-000420	3011790-001014
99401-000421	3011790-001028
99401-000422	3011790-000133
99401-000423	3011790-000143
99401-000424	3011790-000192
99401-000425	3011790-000134
99401-000426	3011790-000148
99401-000427	3011790-000162
99401-000428	3011790-001003
99401-000429	3011790-001008
99401-000430	3011790-001015

Catalog Number	Part Number
99401-000431	3011790-001022
99401-000432	3011790-001043
99401-000433	3011790-000136
99401-000434	3011790-000141
99401-000435	3011790-000144
99401-000436	3011790-000147
99401-000437	3011790-000156
99401-000438	3011790-000159
99401-000439	3011790-000166
99401-000440	3011790-000175
99401-000441	3011790-001009
99401-000442	3011790-000138
99401-000443	3011790-000196
99401-000444	3011790-000163
99401-000445	3011790-000140
99401-000446	3011790-000185
99401-000447	3011790-000149
99401-000448	3011790-000150
99401-000449	3011790-000153
99401-000450	3011790-000176
99401-000451	3011790-000152
99401-000452	3011790-000154
99401-000453	3011790-000177
99401-000454	3011790-000180
99401-000455	3011790-001031
99401-000456	3011790-001035
99401-000457	3011790-000158
99401-000458	3011790-000167
99401-000459	3011790-000174
99401-000460	3011790-001011
99401-000461	3011790-001024
99401-000462	3011790-000161
99401-000463	3011790-000170
99401-000464	3011790-000164
99401-000465	3011790-001001
99401-000466	3011790-001034
99401-000467	3011790-000168
99401-000468	3011790-000171
99401-000469	3011790-001004
99401-000470	3011790-001013
99401-000471	3011790-001018
99401-000472	3011790-000173

Catalog Number	Part Number
99401-000473	3011790-000178
99401-000474	3011790-000191
99401-000475	3011790-000194
99401-000476	3011790-001026
99401-000477	3011790-001036
99401-000478	3011790-000182
99401-000479	3011790-000193
99401-000480	3011790-001023
99401-000481	3011790-001000
99401-000482	3011790-001002
99401-000483	3011790-001006
99401-000484	3011790-001007
99401-000485	3011790-001027
99401-000486	3011790-001037
99401-000487	3011790-001012
99401-000488	3011790-001016
99401-000489	3011790-001042
99401-000490	3011790-001025
99401-000491	3011790-001029
99401-000492	3011790-001030
99401-000493	3011790-001032
99401-000494	3011790-001033
99401-000495	3011790-001038
99401-000496	3011790-001039
99401-000497	3011790-001040
99401-000498	3011790-001041
99401-000499	3011790-001045
99401-000500	3005400-000150
99401-000501	3005400-000151
99401-000502	3011790-001046
99401-000503	3011790-001047
99401-000504	3011790-001048
99401-000505	3011790-001049
99401-000507	3011790-001050
99401-000508	3011790-001051
99401-000509	3011790-001052
99401-000510	3011790-001053
99401-000511	3011790-001054
99401-000524	3011790-001059
99401-000525	3011790-001060
99401-000528	3011790-001063
99401-000529	3011790-001064

Catalog Number	Part Number
99401-000531	3011790-001065
99401-000535	3005400-000155
99401-000539	3005400-000158
99401-000540	3005400-000159
99401-000546	3011790-001075
99401-000549	3005400-000162
99401-000550	3005400-000163
99401-000551	3011790-001077
99401-000552	3011790-001078
99401-000553	3011790-001079
99401-000554	3011790-001080
99401-000559	3011790-001083
99401-000560	3011790-001084
99401-000561	3011790-001085
99401-000562	3011790-001086
99401-000563	3011790-001087
99401-000564	3005400-000167
99401-000565	3005400-000168
99401-000566	3011790-001088
99401-000568	3011790-001090
99401-000569	3005400-000169
99401-000571	3011790-001092
99401-000572	3011790-001093
99401-000573	3011790-001094
99401-000574	3011790-001095
99401-000575	3011790-001096
99401-000576	3011790-001097
99401-000577	3011790-001098
99401-000578	3005400-000170
99401-000579	3011790-001099
99401-000580	3011790-001100
99401-000581	3011790-001101
99401-000582	3011790-001102
99401-000583	3011790-001103
99401-000584	3011790-001104
99401-000585	3011790-001105
99401-000586	3011790-001106
99401-000587	3011790-001107
99401-000588	3011790-001108
99401-000589	3005400-000171
99401-000590	3011790-001109
99401-000591	3011790-001110

Catalog Number	Part Number
99401-000592	3011790-001111
99401-000595	3011790-001114
99401-000596	3011790-001115
99401-000597	3005400-000173
99401-000598	3005400-000174
99401-000600	3011790-001116
99401-000601	3011790-001117
99401-000603	3011790-001118
99401-000604	3005400-000177
99401-000607	3011790-001121
99401-000608	3011790-001122
99401-000609	3005400-000178
99401-000610	3011790-001123
99401-000611	3011790-001124
99401-000612	3011790-001125
99401-000614	3005400-000179
99401-000615	3005400-000180
99401-000617	3011790-001128
99401-000618	3011790-001129
99401-000620	3011790-001131
99401-000621	3011790-001132
99401-000622	3011790-001133
99401-000629	3011790-001140
99401-000633	3011790-001144
99401-000639	3011790-001150
99401-000640	3011790-001151
99401-000641	3011790-001152
99401-000642	3011790-001153
99401-000643	3011790-001154
99401-000644	3011790-001155
99401-000645	3011790-001156
99401-000646	3011790-001157
99401-000648	3011790-001159
99401-000652	3011790-001163
99401-000653	3011790-001164
99401-000654	3011790-001165
99401-000656	3011790-001167
99401-000657	3011790-001168
99401-000660	3011790-001171
99401-000662	3011790-001173
99401-000663	3011790-001174
99401-000667	3011790-001178

Catalog Number	Part Number
99401-000668	3011790-001179
99401-000669	3011790-001180
99401-000671	3011790-001182
99401-000672	3011790-001183
99401-000674	3011790-001185
99401-000675	3011790-001186
99401-000677	3011790-001188
99401-000680	3011790-001191
99401-000682	3011790-001193
99401-000683	3011790-001194
99401-000685	3011790-001196
99401-000689	3011790-001200
99401-000692	3011790-001203
99401-000693	3011790-001204
99401-000696	3011790-001207
99401-000697	3011790-001208
99401-000699	3011790-001210
99401-000700	3011790-001211
99401-000702	3011790-001213
99401-000704	3011790-001215
99401-000706	3011790-001217
99401-000708	3011790-001219
99401-000710	3011790-001221
99401-000711	3011790-001222
99401-000714	3011790-001225
99401-000716	3011790-001227
99401-000717	3011790-001228
99401-000719	3011790-001230
99401-000724	3011790-001235
99401-000729	3011790-001240
99401-000730	3011790-001241
99401-000733	3011790-001244
99401-000744	3011790-001255
99401-000746	3011790-001257
99401-000748	3011790-001259
99401-000749	3011790-001260
99401-000753	3011790-001264
99401-000754	3011790-001265
99401-000757	3011790-001268
99401-000763	3011790-001274
99401-000773	3011790-001284
99401-000774	3011790-001285

Catalog Number	Part Number
99401-000775	3011790-001286
99401-000776	3011790-001287
99401-000782	3011790-001293
99401-000784	3011790-001295
99401-000785	3011790-001296
99401-000788	3011790-001299
99401-000790	3011790-001301
99401-000796	3011790-001307
99401-000799	3011790-001310
99401-000801	3011790-001312
99401-000803	3011790-001314
99401-000804	3011790-001315
99401-000805	3011790-001316
99401-000806	3011790-001317
99401-000807	3011790-001318
99401-000808	3011790-001319
99401-000809	3011790-001320
99401-000810	3011790-001321
99401-000811	3011790-001322
99401-000812	3011790-001323
99401-000813	3011790-001324
99401-000814	3011790-001325
99401-000815	3011790-001326
99401-000818	3011790-001329
99401-000823	3005400-000181
99401-000824	3005400-000182
99401-000825	3011790-001334
99401-000826	3011790-001335
99401-000827	3011790-001336
99401-000829	3011790-001337
99401-000830	3011790-001338
99401-000836	3011790-001340
99401-000837	3011790-001341
99401-000838	3011790-001342
99401-000839	3011790-001343
99401-000840	3011790-001344
99401-000842	3011790-001346
99401-000843	3005400-000185
99401-000845	3011790-001348
99401-000846	3005400-000187
99401-000848	3005400-000188
99401-000849	3011790-001350

Catalog Number	Part Number
99401-000850	3005400-000189
99401-000851	3011790-001351
99401-000852	3011790-001352
99401-000853	3005400-000190
99401-000854	3011790-001353
99401-000855	3011790-001354
99401-000856	3011790-001355
99401-000857	3005400-000191
99401-000858	3011790-001356
99401-000865	3011790-001358
99401-000866	3011790-001359
99401-000868	3011790-001361
99401-000869	3011790-001362
99401-000872	3011790-001363
99401-000875	3011790-001366
99401-000876	3011790-001367
99401-000877	3005400-000199
99401-000878	3011790-001368
99401-000879	3005400-000200
99401-000880	3011790-001369
99401-000881	3011790-001370
99401-000882	3011790-001371
99401-000883	3011790-001372
99401-000885	3011790-001374
99401-000886	3011790-001375
99401-000887	3011790-001376
99401-000889	3011790-001378
99401-000890	3011790-001379
99401-000891	3011790-001380
99401-000892	3011790-001381
99401-000893	3011790-001382
99401-000895	3011790-001383
99401-000897	3011790-001385
99401-000898	3011790-001386
99401-000901	3011790-001389
99401-000903	3011790-001391
99401-000904	3011790-001392
99401-000905	3011790-001393
99401-000908	3011790-001396
99401-000909	3011790-001397
99401-000910	3011790-001398
99401-000912	3011790-001400

Catalog Number	Part Number
99401-000913	3011790-001401
99401-000914	3011790-001402
99401-000916	3011790-001404
99401-000917	3011790-001405
99401-000920	3011790-001408
99401-000923	3011790-001411
99401-000924	3011790-001412
99401-000926	3011790-001414
99401-000927	3011790-001415
99401-000928	3011790-001416
99401-000929	3011790-001417
99401-000930	3011790-001418
99401-000934	3011790-001422
99401-000941	3011790-001429
99401-000945	3011790-001433
99401-000947	3011790-001434
99401-000948	3011790-001435
99401-000949	3005400-000203
99401-000950	3011790-001436
99401-000951	3011790-001437
99401-000952	3011790-001438
99401-000955	3011790-001441
99401-000956	3011790-001442
99401-000960	3011790-001446
99401-000962	3011790-001448
99401-000963	3011790-001449
99401-000966	3011790-001452
99401-000967	3011790-001453
99401-000968	3011790-001454
99401-000969	3011790-001455
99401-000970	3011790-001456
99401-000971	3011790-001457
99401-000974	3011790-001460
99401-000975	3011790-001461
99401-000978	3011790-001464
99401-000979	3011790-001465
99401-000980	3011790-001466
99401-000981	3011790-001467
99401-000982	3011790-001468
99401-000984	3011790-001470
99401-000986	3011790-001472
99401-000987	3011790-001473

Catalog Number	Part Number
99401-000988	3011790-001474
99401-000991	3011790-001477
99401-000992	3011790-001478
99401-000993	3011790-001479
99401-000994	3011790-001480
99401-000995	3011790-001481
99401-001013	3011790-001499
99401-001014	3011790-001500
99401-001015	3011790-001501
99401-001016	3011790-001502
99401-001017	3011790-001503
99401-001018	3011790-001504
99401-001019	3011790-001505
99401-001020	3011790-001506
99401-001021	3011790-001507
99401-001022	3011790-001508
99401-001023	3011790-001509
99401-001024	3011790-001510
99401-001025	3011790-001511
99401-001026	3011790-001512
99401-001027	3011790-001513
99401-001028	3011790-001514
99401-001029	3011790-001515
99401-001030	3011790-001516
99401-001031	3011790-001517
99401-001032	3011790-001518
99401-001033	3011790-001519
99401-001034	3011790-001520
99401-001035	3011790-001521
99401-001036	3011790-001522
99401-001037	3011790-001523
99401-001038	3011790-001524
99401-001040	3011790-001526
99401-001043	3011790-001529
99401-001044	3011790-001530
99401-001045	3011790-001531
99401-001046	3011790-001532
99401-001051	3011790-001537
99401-001053	3011790-001539
99401-001054	3011790-001540
99401-001055	3011790-001541
99401-001058	3011790-001544

Catalog Number	Part Number
99401-001060	3011790-001546
99401-001063	3011790-001549
99401-001065	3011790-001551
99401-001066	3011790-001552
99401-001067	3011790-001553
99401-001068	3011790-001554
99401-001069	3011790-001555
99401-001070	3011790-001556
99401-001071	3011790-001557
99401-001077	3011790-001563
99401-001078	3011790-001564
99401-001079	3011790-001565
99401-001081	3011790-001567
99401-001082	3011790-001568
99401-001083	3011790-001569
99401-001084	3011790-001570
99401-001085	3011790-001571
99401-001086	3011790-001572
99401-001087	3011790-001573
99401-001088	3011790-001574
99401-001089	3011790-001575
99401-001090	3011790-001576
99401-001091	3011790-001577

Catalog Number	Part Number
99401-001093	3011790-001579
99401-001098	3011790-001584
99401-001099	3011790-001585
99401-001100	3011790-001586
99401-001103	3011790-001589
99401-001107	3011790-001593
99401-001110	3011790-001596
99401-001111	3011790-001597
99401-001112	3011790-001598
99401-001116	3011790-001602
99401-001117	3011790-001603
99401-001118	3011790-001604
99401-001119	3011790-001605
99401-001120	3011790-001606
99401-001121	3011790-001607
99401-001122	3011790-001608
99401-001125	3011790-001611
99401-001126	3011790-001612
99401-001128	3011790-001614
99401-001131	3011790-001617
99401-001133	3011790-001619
99401-001134	3011790-001620
99401-001135	3011790-001621

Catalog Number	Part Number
99401-001136	3011790-001622
99401-001137	3011790-001623
99401-001138	3011790-001624
99401-001139	3011790-001625
99401-001140	3011790-001626
99401-001141	3011790-001627
99401-001148	3011790-001634
99401-001151	3011790-001637
99401-001154	3011790-001640
99401-001155	3011790-001641
99401-001160	3011790-001646
99401-001161	3011790-001647
99401-001162	3011790-001648
99401-001163	3011790-001649
99401-001164	3011790-001650
99401-001165	3011790-001651
99401-001167	3011790-001653
99401-001170	3011790-001656
99401-001172	3011790-001658
99401-001173	3011790-001659
99401-001183	3011790-001669
-	3011790
-	3011790-000002