SERO AS

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Product: Pathonorm™ H
Article no.: 100705
FSN ID: 2016-02
Action: Information

Date issued: 18/4-16

Attention: SERO AS Distributor /SERO AS Customer

Details of affected device:

Product Name: Pathonorm™ H
Article no.: 100705
Lot number: 1403120

Description of the problem

The component phenylalanine has been present in the above mentioned product as an added amount since 1960s. No analytical data has been presented for this component and the product has not been intended to be used as a trueness control for phenylalanine. SERO has not marketed the product as a control for analysis used in the diagnosis of phenylketonuria and the product has been CE-marked following the conformity assessment procedure for general IVD-products.

SERO has many CE-marked IVD- products since the introduction of the IVD-directive, including also list II B products. During a routine audit of SERO's Quality system from the Notified Body VTT (10.3.2016) concerning products CE-marked with the approval of VTT in accordance with List IIB requirements, a complete review of all SERO products was performed. This included a review of the composition, assigned values and stated intended use. During the audit, it was brought to our attention that customers may potentially use the product Pathonorm™ H as a precision control for analysis used for the diagnosis of phenylketonuria and that the product therefore should have been considered as a candidate for the approval of a Notified Body as stated in list II B of the IVD-directive. To ensure that the product is not used for the control of analysis intended for the diagnosis of phenylketonuria, we have taken the decision to delete the information about the added amount of this component in the lot-specific product documentation.

The version number of the updated lot-specific product documentation is as follows:

- Pathonorm H lot # 1403120 2016-04

There will be no reference to this component in any other promotion or information material concerning the product.

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Necessary actions to be taken by the distributor / user:

Please inform your customers by distributing this letter to all users of the above mentioned product. New updated lot-specific product documentation is available in the SERO library online (www.sero.no/theserolibrary). A copy of the updated lot-specific product documentation for the concerned lots is attached with this field safety notice.

Any reportable incidents related to the use of this product as a control for analysis intended for the diagnosis of phenylketonuria should be reported to SERO.

Risk or patient impact

Negligible.

Recommended patient follow up:

To this date, SERO has not received any reports of incidents affecting patients related to the use of the product upon diagnosis of phenylketonuria.

Each laboratory must consider the need for patient follow up depending on their use of the product.

Transmission of this Field Safety notice:

This notice needs to be passed on to all those who need to be aware within your organization or to any organization where Pathonorm™ H lot no. 1403120 has been transferred.

Please transfer this field safety notice to other organizations on which this action has an impact (if appropriate).

Contact information

If you have any questions regarding this information, please contact our Customer Service (+47 66 85 89 00).

We kindly ask for your understanding for this information and thank you for your cooperation.

Sincerely

Silje Solheim Johnsen
Head of Product Management

Åse Davanger
Quality Manager

REF 100705

LOT 1403120

 2018-04

10 x 5 mL

IVD

CE

Collaborating laboratories

The analytical data of Pathonorm™ H have been elaborated in collaboration with the following independent laboratories:

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- The Laboratory of SERO AS, Billingstad, Norway
- Department of Medical Biochemistry, St.Olav's Hospital, Trondheim, Norway

EN

Storage and stability

This product will be stable until the expiry date when stored unopened at 2-8 °C.

After reconstitution, all analytes are stable for 7 days when stored tightly capped at 2-8 °C.

Vials to be aliquoted and frozen within 30 minutes are stable 1 month at ≤ -20 °C.

Limitations

All stability data require that bacterial contamination is avoided. Increased turbidity may indicate bacterial growth.

CK is sensitive to temperature changes.
CK and Bilirubin are sensitive to light.

Assignment of values

The mean analytical values and acceptable ranges given in the table below are derived from replicate analyses obtained through collaboration with a number of independent laboratories and they are specific for this lot. The uncertainty of the analytical value is presented as a single number, U (with a coverage factor $k = 2$), which takes into account various factors including imprecision of the method applied for the assignment as well as the stability of the given component. Individual laboratory means should fall within the acceptable range. However, variations in the analytical value and acceptable range for each component may vary over time due to differences in reagent and/or calibrator lots as well as the instrument and/or method used. It is therefore recommended that each laboratory establishes its own analytical mean and acceptable range and only use the provided values as a guide. If applicable, the Rilibäk range (Richtlinien der Bundesärztekammer zur Qualitätssicherung in medizinischen Laboratorien (2008)) is provided.

The value assignment has been established in accordance with the Essential Requirements (Annex 1 of the IVD Directive) 98/79/EC, and the ISO 17511²) International standard.

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1) Council Directive 98/79/EC of the European Parliament and the Council of 27 October 1998 on In Vitro Diagnostic Medical Devices (Annex 1: Essential requirements).

2) ISO 17511. In Vitro Diagnostic Medical Devices- Measurement of quantities in samples of biological origin- Metrological traceability of values assigned to calibrators and control materials. Geneva, International Organisation for Standardization 2003.

3) This is not a Rilibak range, but SERO's assessment

4) To avoid Zinc contamination from vial and rubber stopper, transfer the serum to plastic tubes after reconstitution