

Siemens Healthcare GmbH, SHS DI XP,  
Siemensstr. 3, Healthcare Building 1, 91301 Forchheim

Name Volker Biermann  
Abteilung SHS DI XP M&S-M IN TRUMF

Telefon +49 (9191) 18-4079

E-Mail [volker.j.biermann@siemens-healthineers.com](mailto:volker.j.biermann@siemens-healthineers.com)  
Datum April 11th 2019

## **FIELD SAFETY NOTICE**

Important safety information for customers regarding a field corrective action

**XP011/18/S, XP023/18/S, XP025/18/S and XP031/18/S**

To all users of systems

AXIOM Vertex Solitaire M / MD Trauma, AXIOM MULTIX M / TOP / Pro, AXIOM Aristos MX/VX, Multix Fusion, Ysio (Max), Sireskop SD, Iconos R200, LuminosTF, Luminos Agile (Max), Luminos dRF (Max), Luminos Fusion

Dear customer,

This letter is to inform about a preventive action that was performed on the above mentioned systems. The tube stand driveline inside the lifting column was reworked with a modified version.

### **What is the modification of the tube stand driveline?**

The modified tube stand driveline provides an improved mechanism to detect deterioration of the steel cables.

### **What are the possible risks to health?**

To prevent a potential risk caused by the deterioration of the steel cables, the related component was reworked preventively. We are not aware of any incidents regarding the above-referenced systems. There is no risk to health related to operation of your system.

Siemens Healthcare GmbH  
Management: Bernhard Montag, Chairman;  
Jochen Schmitz, Michael Reitermann

Siemensstr. 3  
Healthcare Building 1  
91301 Forchheim  
Germany

[siemens.com/healthcare](http://siemens.com/healthcare)

Chairman of the Supervisory Board: Michael Sen  
Registered office: Munich, Germany; Commercial Registry: Munich, HRB 213821  
WEEE-Reg.-No. DE 64872105

**When could the problem occur?**

A problem with the steel cables could, depending on the degree of usage, have come up during the ongoing lifetime of a system. The possibility of occurrence was rated as extremely rare.

The following systems representing a potential risk due to high usage have already been updated:

|  |            |
|--|------------|
| Axiom Aristos Family                         | XP011/18/S |
| Multix/Vertex Family                         | XP025/18/S |
| Ysio, Luminos Agile, Luminos dRF, Max Family | XP023/18/S |
| Multix Fusion, Luminos Fusion                | XP031/18/S |

**How was the field action realized?**

– This letter informs that the activities outlines in the above mentioned updates are executed preventively at your site. A decision was made to perform this action under the IDs of the previous field corrective actions.

**What if you no longer have this device/equipment?**

If this device/equipment is no longer in your possession, please forward this safety notice to the new owner of this device/equipment. If applicable, please inform us about the new owner of the device/equipment.

Sincerely Yours



Carsten Bertram  
Head of SHS DI XP



Christian Denger  
Head of SHS DI XP QT