

URGENT Field Safety Notice

FSN: 2022-CC-SRC-001

**Philips Respironics – Sleep and Respiratory Care
Notification to Patients**

**Amara View Minimal Contact Full-Face Mask
DreamWear Full Face Mask
DreamWisp Nasal Mask with Over the Nose Cushion
Wisp Nasal Masks and Wisp Youth Nasal Masks
Therapy Mask 3100 NC/SP**

This document contains important information for the continued safe and proper use of your equipment

- Please review the following information with all members of your staff who need to be aware of the contents of this communication. It is important to understand the implications of this communication.
- Please retain a copy with the equipment Instruction for Use.

August XX, 2022

Dear Patient,

The following Philips Respironics patient interface devices (face and nasal masks) - Amara View Minimal Contact Full-Face Mask, DreamWear Full Face Mask, DreamWisp Nasal Mask with Over the Nose Cushion, Wisp Nasal Masks and Wisp Youth Nasal Masks, and Therapy Mask 3100 NC/SP - contain magnets.

If you do not have implanted medical devices, or metallic splinters in your eyes, then no action is needed.

Philips Respironics is updating its existing contraindications and warnings to the below. If you have any type of active implantable devices (or metallic splinters in your eyes) and are using any of these masks with magnets, please ensure that you read and understand the updated contraindication and warning:

Contraindication: *Use of the mask is contraindicated in patients with metallic splinters in the eye and the following metallic implants: brain shunts, aneurysm clips, pacemakers, implantable cardioverter defibrillators (ICD), defibrillators, VP shunts, and certain neurostimulators used in and around the neck.*

Warning: *Magnets with a magnetic field strength of 400mT are used in the mask. With the exception of the devices identified in the contraindication, ensure the mask is kept at least 2 inches (5 cm) away from any other active medical implant or medical device that can be impacted by the magnetic fields to avoid possible effects from localized magnetic fields.*

Note that the current labeling for these devices already have warnings related to magnets. The updated contraindications and warnings strengthen this.

1. What the problem is and under what circumstances it can occur

The affected masks contain magnets which can potentially affect the functioning of certain implanted medical devices, such as metal splinters in the eyes, brain shunts, aneurysm clips, pacemakers, implantable cardioverter defibrillators (ICD), defibrillators, VP shunts, and certain neurostimulators used in and around the neck, or any implanted medical device affected by magnets.

2. Describe the hazard/harm associated with the issue

With the exception of the devices identified in the contraindication, if the mask magnets are placed less than 2 inches away from an implanted metallic device the magnets may cause the device to not perform as intended, which may result in a serious injury. There have been nine (9) reports of serious injuries associated with this issue.

3. Affected products and how to identify them

Images of the affected masks are shown below. The magnetic headgear clips on these masks are circled. These images may be used as reference to determine if your or your patient's mask also uses magnetic headgear clips. These masks are intended to provide an interface for application of CPAP or bi-level therapy to patients. The part numbers for the affected masks and mask components that contain magnets are listed in Figure 6.



Figure 1:
Amara View
Full Face
Mask



Figure 2:
DreamWisp
Nasal Mask



Figure 3:
DreamWear
Full Face
Mask



Figure 4:
Wisp and
Wisp Youth
Nasal Mask



Figure 5:
Therapy
Mask 3100
NC/SP

4. Describe the actions that should be taken by the patient in order to prevent risks

As a patient using these masks, you must take the following action:

- 4.1. If you do not have implanted medical devices, or metallic splinters in your eyes, then no action is needed.
- 4.2. If you have implanted medical devices that may interact with magnets and use one of the affected masks identified in this notice it is important to be aware of the potential interaction between the masks and your device(s) as noted in the contraindication/warning listed above. You should consult your physician to determine if another mask should be used for your therapy. Philips Respironics customer service can be contacted to learn more about non-magnetic mask options.
- 4.3. If you experience any of the issues discussed above, contact a healthcare provider immediately.

5. Describe the actions planned by Philips to correct the problem

Philips Respironics is updating its existing contraindication and warning as provided in this field safety notice.

6. If you need any further information or support concerning this issue, please contact your local Philips Respironics representative.

This notice has been reported to the appropriate Regulatory Agencies. Philips Respironics regrets any inconvenience caused by this problem.

Sincerely,

Thomas Fallon
 Head of Quality Assurance
 Philips Respiroics

Devices Impacted by This Issue - Figure 6:

| Amara View Minimal Contact Full-Face Mask | |
|--|-------------------------------------|
| Part # | Description |
| 1090602 | S Amara View Mask w/HGR |
| 1090603 | M Amara View Mask w/HGR |
| 1090604 | L Amara View Mask w/HGR |
| 1090622 | S Amara View Mask w/ HGR, DOM |
| 1090623 | M Amara View Mask w/ HGR, DOM |
| 1090624 | L Amara View Mask w/ HGR, DOM |
| 1090641 | XS Amara View Mask w/HGR, JPN |
| 1090642 | S Amara View Mask w/HGR, JPN |
| 1090643 | M Amara View Mask w/HGR, JPN |
| 1090644 | L Amara View Mask w/HGR, JPN |
| 1090651 | XS Amara View Mask SE w/HGR, JPN |
| 1090652 | S Amara View Mask SE w/HGR, JPN |
| 1090653 | M Amara View Mask SE w/HGR, JPN |
| 1090654 | L Amara View Mask SE w/HGR, JPN |
| 1090662 | S Amara View Mask w/HGR, INTL |
| 1090663 | M Amara View Mask w/HGR, INTL |
| 1090664 | L Amara View Mask w/HGR, INTL |
| 1090670 | Amara View Mask w/HGR, FitPack |
| 1090696 | RP – Amara View Headgear, L |
| 1090697 | RP-Amara View Headgear, Standard |
| 1135080 | S AMARA VIEW MASK SE W/ HGR |
| 1135081 | M AMARA VIEW MASK SE W/ HGR |
| 1135082 | L AMARA VIEW MASK SE W/ HGR |
| 1136626 | Amara View Magnet Clip |
| 1090670RTL | AMARA VIEW MASK W/HGR, FITPACK-RTL |
| 1136626RTL | Amara View Full Face Magnetic Clips |

| DreamWear Full Face Mask | |
|---------------------------------|---------------------------------------|
| Part # | Description |
| 1133344 | S, DreamWear UTN Nsl/Full, Combo Pack |
| 1133345 | M, DreamWear UTN Nsl/Full, Combo Pack |
| 1133370 | S, DreamWear Full, Sm & Med Frm, INTL |
| 1133371 | M, DreamWear Full, Sm & Med Frm, INTL |
| 1133372 | L, DreamWear Full, Sm & Med Frm, INTL |
| 1133373 | MW, DreamWear Full, Sm & Med Frm |

| Wisp and Wisp Youth Nasal Mask | |
|---------------------------------------|--|
| Part # | Description |
| 1094050 | WISP MASK, CLEAR FRAME, W/ HDGR, DOM |
| 1094051 | WISP MASK, FABRIC FRAME, W/MAG CLIPS DOM |
| 1094053 | WISP MASK, USA, FABRIC FRAME, W/ HDGR |
| 1094060 | WISP MASK, CLEAR FRAME, W/ HDGR, INTL |
| 1094061 | WISP MASK, FABRIC FRAME, W/ HDGR, INTL |
| 1094062 | WISP MASK, CLEAR FRAME, W/ HDGR, INTL SE |
| 1094070 | WISP MASK, CLEAR FRAME, W/ HDGR, JPN SE |
| 1094071 | P, WISP MASK, FABRIC FRAME, W/ HDGR, JPN |
| 1094072 | S/M, WISP MASK FABRIC FRAME, W/ HDGR JPN |
| 1094073 | L, WISP MASK, FABRIC FRAME, W/ HDGR, JPN |
| 1109298 | WISP YOUTH MASK, FABRIC, W/MAG CLIPS DOM |
| 1113954 | XL, Wisp Mask, Fabric Frame, w/ HDGR, JP |
| 1117119 | MW, Wisp Mask, Fabric Frame, w/ HDGR, JP |
| 1118060 | P, WISP MASK, CLEAR FRAME, W/ HDGR, DOM |
| 1118061 | S/M, WISP MASK CLEAR FRAME, W/ HDGR DOM |
| 1118062 | L, WISP MASK, CLEAR FRAME, W/ HDGR, DOM |
| 1118063 | XL, WISP MASK, CLEAR FRAME, W/ HGR, DOM |
| 1118064 | P, WISP MASK, FABRIC FRAME, W/ HDGR,DOM |
| 1118065 | S/M, WISP MASK FABRIC FRAME W/ HDGR,DOM |
| 1118066 | L, WISP MASK, FABRIC FRAME, W/ HDGR,DOM |
| 1118067 | XL, WISP MASK, FABRIC FRAME W/ HGR, DOM |
| 1120434 | Wisp/DreamWisp Magnetic Headgear Clip,RP |
| 1137360 | Wisp Nasal Lab Mask, Fabric Frame, DOM |
| 1137361 | Wisp Nasal Lab Mask, Clear Frame, DOM |
| 1137366 | S/M, Wisp Lab Mask, Fabric Frame, DOM |
| 1137367 | L, Wisp Nasal Mask, Fabric Frame, DOM |
| 1137368 | S/M, Wisp Nasal Mask, Clear Frame, DOM |
| 1137369 | L, Wisp Nasal Mask, Clear Frame, DOM |
| 1137370 | COMBOPACK, WISP LAB MASK ,DOM |
| 1138989 | P, WISP MASK,CLEAR FRAME, W/HDGR,1118060 |
| 1138991 | L, WISP MASK, CLEAR W/ HDGR, DOM 1118062 |
| 1138992 | XL,WISP MASK,CLEAR FRAME,W/ HGR,1118063 |
| 1138993 | P, WISP MASK,FABRIC FRAME,W/HDGR,1118064 |
| 1138994 | S/M,WISP MASK FABRICFRAME W/HDGR,1118065 |
| 1138995 | L, WISP MASK,FABRIC FRAME,W/HDGR,1118066 |
| 1138996 | XL,WISP MASK,FABRIC FRAME W/HGR, 1118067 |

| | |
|---------|--|
| 1133375 | S, DreamWear Full, Sm & Med Frm, GBL |
| 1133376 | M, DreamWear Full, Sm & Med Frm, GBL |
| 1133377 | L, DreamWear Full, Sm & Med Frm, GBL |
| 1133378 | MW, DreamWear Full, S. & M. Frm, GBL |
| 1133380 | S, DreamWear Full, M. Frm W/ HGR, GBL |
| 1133381 | M, DreamWear Full, Med Frm W/ HGR |
| 1133382 | L, DreamWear Full, Med Frm W/ HGR |
| 1133383 | MW, DreamWear Full, Med Frm W/ HGR |
| 1133385 | S, DreamWear Full, Sm Frm W/ HGR, GBL |
| 1133386 | M, DreamWear Full, Sm Frm W/ HGR |
| 1133387 | L, DreamWear Full, Sm Frm W/ HGR, GBL |
| 1133388 | MW, DreamWear Full, Sm Frm W/ HGR, |
| 1133390 | S, DreamWear Full, Lg Frm W/ HGR, GBL |
| 1133391 | M, DreamWear Full, Lg Frm W/ HGR, GBL |
| 1133392 | L, DreamWear Full, Lg Frm W/ HGR, GBL |
| 1133393 | MW, DreamWear Full, Lg Frm W/ HGR |
| 1133395 | DreamWear Full, Demo Pack |
| 1133400 | FitPack, DreamWear Full, Med Frame |
| 1133401 | DreamWear Full, Med Frm W/O HGR, GBL |
| 1133405 | S, DreamWear Full, Med Frm W/O HGR |
| 1133406 | M, DreamWear Full, Med Frm W/O HGR |
| 1133407 | L, DreamWear Full, Med Frm W/O HGR |
| 1133408 | MW, DreamWear Full, Med Frm W/O |
| 1133410 | S, DreamWear Full, Sm Frm W/O HGR |
| 1133411 | M, DreamWear Full, Sm Frm W/O HGR |
| 1133412 | L, DreamWear Full, Sm Frm W/O HGR |
| 1133413 | MW, DreamWear Full, Sm Frm W/O HGR |
| 1133415 | S, DreamWear Full, Lg Frm W/O HGR, GBL |
| 1133416 | M, DreamWear Full, Lg Frm W/O HGR |
| 1133417 | L, DreamWear Full, Lg Frm W/O HGR, GBL |
| 1133418 | MW, DreamWear Full, Lg Frm W/O HGR |
| 1133423 | S/M, DreamWear Full, Sm Frm w/ HGR |
| 1133424 | S/M, DreamWear Full, Med Frm w/ HGR |
| 1133425 | S/M, DreamWear Full, Lg Frm w/ HGR |
| 1133427 | W, DreamWear Full, Med Frm W/ HGR |
| 1133430 | S, DreamWear Full Cushion, RP |
| 1133431 | M, DreamWear Full Cushion, RP |
| 1133432 | L, DreamWear Full Cushion, RP |
| 1133433 | MW, DreamWear Full Cushion, RP |
| 1133434 | SW, DreamWear Full Cushion, RP |
| 1133435 | S, DreamWear Full Cushion, RP, JPN |
| 1133436 | M, DreamWear Full Cushion, RP, JPN |
| 1133438 | W, DreamWear Full Cushion, RP, JPN |
| 1133450 | DreamWear Full Headgear, RP |
| 1133454 | Headgear Magnet Clip, RP |
| 1135275 | S, DreamWear Full, Sm and Med Frm |
| 1135276 | M, DreamWear Full, Sm and Med Frm |

| | |
|------------|--|
| 1139009 | S/M,WISPMASK CLEAR FRAME,W/ HDGR 1118061 |
| 1094051RTL | WISP MASK, FABRICFRM,W/MAG CLIPS DOM-RTL |
| 1148366 | DreamWisp/Wisp Magnetic Clips - RTL |

| DreamWisp Nasal Mask with Over the Nose Cushion | |
|--|--|
| Part # | Description |
| 1137916 | Fitpk DreamWisp Nsl, MED Con W/HGR GBL |
| 1137918 | FitPk DreamWisp Nsl, MED Con W/O HGR GBL |
| 1137921 | FitPk, DreamWisp Nsl, MED Con W/ HGR INT |
| 1137922 | P, DreamWisp Nsl, Med Con W/O HGR GBL |
| 1137923 | S, DreamWisp Nsl, Med Con W/O HGR GBL |
| 1137924 | M, DreamWisp Nsl, Med Conn W/O HGR GBL |
| 1137925 | L, DreamWisp Nsl, Med Con W/O HGR GBL |
| 1137926 | XL, DreamWisp Nsl, Med Con W/O HGR GBL |
| 1137927 | P, DreamWisp NSL, SM CON, W/HGR, GBL |
| 1137928 | S, DreamWisp NSL, SM CON, W/HGR, GBL |
| 1137929 | M, DreamWisp NSL, SM CON, W/HGR, GBL |
| 1137930 | L, DreamWisp NSL, SM CON, W/HGR, GBL |
| 1137931 | XL, DreamWisp NSL, SM CON, W/HGR, GBL |
| 1137932 | P, DreamWisp Nsl, Med Con, W/HGR, GBL |
| 1137933 | S, DreamWisp Nsl, Med Con, W/HGR, GBL |
| 1137934 | M, DreamWisp Nsl, Med Con, W/HGR, GBL |
| 1137935 | L, DreamWisp Nsl, Med Con, W/HGR, GBL |
| 1137936 | XL, DreamWisp Nsl, Med Con, W/HGR, GBL |
| 1137937 | P, DreamWisp NSL, SM CON, W/HGR, INT |
| 1137938 | S, DreamWisp NSL, SM CON, W/HGR, INT |
| 1137939 | M, DreamWisp NSL, SM CON, W/HGR, INT |
| 1137940 | L, DreamWisp NSL, SM CON, W/HGR, INT |
| 1137941 | XL, DreamWisp NSL, SM CON, W/HGR, INT |
| 1137942 | P, DreamWisp Nsl, Med Con, W/HGR INT |
| 1137943 | S, DreamWisp Nsl, Med Con, W/HGR INT |
| 1137944 | M, DreamWisp Nsl, Med Con, W/HGR INT |
| 1137945 | L, DreamWisp Nsl, Med Con, W/HGR INT |
| 1137946 | XL, DreamWisp Nsl, Med Con, W/HGR INT |
| 1137947 | P, DreamWisp Nsl, S-M-L Con, W/HGR, JPN |
| 1137948 | S, DreamWisp Nsl, S-M-L Con, W/HGR, JPN |
| 1137949 | M, DreamWisp Nsl, S-M-L Con, W/HGR, JPN |
| 1137950 | L, DreamWisp Nsl, S-M-L Con, W/HGR, JPN |
| 1137951 | XL, DreamWisp Nsl, S-M-L Con, W/HGR, JPN |
| 1137953 | LABPK DreamWisp ALL CON S/M/L CUSH W/HGR |
| 1137954 | P/XL LABPK DreamWisp LAB PACK, S/L con |
| 1137955 | Combo LabPK DreamWisp |
| 1137964 | DreamWisp HEADGEAR, RP |
| 1137965 | DreamWisp LARGE HEADGEAR, RP |
| 1137966 | DreamWisp RS HEADGEAR, RP |
| 1139332 | P, DreamWisp Nsl, S-M Con, W/HGR KR |

| | |
|------------|---------------------------------------|
| 1135277 | SW, DreamWear Full,Sm and Med Frm |
| 1135278 | MW, DreamWear Full,Sm and Med Frm |
| 1135327 | S, DreamWear Full, Sm & Med Frm, KR |
| 1135388 | M, DreamWear Full, Sm & Med Frm, KR |
| 1135389 | SW, DreamWear Full, Sm & Med Frm, KR |
| 1135390 | MW, DreamWear Full, Sm & Med Frm |
| 1137430 | Fitpack, DreamWear Full Lab,Med Frm |
| 1137431 | Ext Sizepack,DreamWear Full Lab Msk |
| 1137437 | S, DreamWear Full Lab Msk,Med Frame |
| 1137438 | S, DreamWear Full Lab Mask,Sm Frame |
| 1137439 | M, DreamWear Full Lab Msk,Med Frame |
| 1137440 | COMBOPACK, DREAMWEAR FULL LAB |
| 1140235 | S, DreamWear Full, Med Frm W/ HGR |
| 1143130 | FitPack, DreamWear Full, Med Frame |
| 1144347 | DreamWear Full Headgear - Lab, RP |
| 1146593 | S, DreamWear Full Cushion, RP - RTL |
| 1146594 | M, DreamWear Full Cushion, RP - RTL |
| 1146595 | L, DreamWear Full Cushion, RP - RTL |
| 1146596 | MW, DreamWear Full Cushion, RP - RTL |
| 1146597 | DreamWear Full Headgear, RP - RTL |
| 1148363 | DreamWear Full Face Magnet Clip - RTL |
| 1133375RTL | DreamWear Full Face Mask Kit, Small |
| 1133376RTL | DreamWear Full Face Mask Kit, Medium |
| 1133377RTL | DreamWear Full Face Mask Kit, Large |
| 1133378RTL | DreamWear Face Mask Kit, M. Cushion |
| 1133430RTL | S, DREAMWEAR FULL CUSHION, RP-RTL |
| 1133431RTL | M, DREAMWEAR FULL CUSHION, RP-RTL |
| 1133432RTL | DreamWear Full Face Cushion, Large |
| 1133433RTL | DreamWear Full Face Cushion, Medium |

| Therapy Mask 3100 NC/SP | |
|--------------------------------|---------------------------------------|
| 1144608 | Therapy Msk 3100 NC, fitpack, w/HGR |
| 1144610 | Therapy Msk 3100 SP, fitpack, w/HGR |
| 1145038 | Therapy Msk 3100 NC, fitpack, w/HGR |
| 1145039 | Therapy Mask 3100 NC, fitpack, w/HGR |
| 1145040 | Therapy Mask 3100 NC, fitpack, w/HGR |
| 1145041 | Therapy Mask 3100 NC, fitpack, w/HGR |
| 1145044 | XS, Therapy Mask 3100 NC, w/oHGR, GBL |
| 1145045 | S, Therapy Mask 3100 NC, w/oHGR, GBL |
| 1145046 | M, Therapy Mask 3100 NC, w/oHGR, GBL |
| 1145047 | L, Therapy Mask 3100 NC, w/oHGR, GBL |
| 1145048 | MW, Therapy Mask 3100 NC, w/oHGR |
| 1145049 | Therapy Msk 3100 SP, fitpack, w/HGR, |
| 1145050 | Therapy Mask 3100 SP, fitpack, w/HGR |
| 1145051 | Therapy Mask 3100 SP, fitpack, w/HGR |
| 1145052 | Therapy Mask 3100 SP, fitpack, w/HGR |

| | |
|------------|---|
| 1139333 | S, DreamWisp Nsl, S-M Con, W/HGR KR |
| 1139334 | M, DreamWisp Nsl, S-M Con, W/HGR KR |
| 1139335 | L, DreamWisp Nsl, S-M Con, W/HGR KR |
| 1139336 | XL, DreamWisp Nsl, S-M Con, W/HGR KR |
| 1144344 | DreamWisp RS HEADGEAR-Lab, RP |
| 1144345 | DreamWisp LARGE HEADGEAR-Lab, RP |
| 1144346 | DreamWisp HEADGEAR - Lab, RP |
| 1144502 | FitPack, DreamWisp, USA |
| 1148409 | DreamWisp Headgear (RS)- RTL |
| 1148410 | DreamWisp HEADGEAR, RP- RTL |
| 1148411 | DreamWisp LARGE HEADGEAR, RP- RTL |
| 1148913 | P, DreamWisp Nsl, S-M Con, W/HGR CN |
| 1148914 | S, DreamWisp Nsl, S-M Con, W/HGR CN |
| 1148915 | M, DreamWisp Nsl, S-M Con, W/HGR CN |
| 1148916 | L, DreamWisp Nsl, S-M Con, W/HGR CN |
| 1148917 | XL, DreamWisp Nsl, S-M Con, W/HGR CN |
| 1137942AP | P, DREAMWISP NSL, MED CON, W/HGR, AP |
| 1137942CE | P, DREAMWISP NSL, MED CON, W/HGR, CE |
| 1137942RC | P, DREAMWISP NSL, MED CON, W/HGR, RC |
| 1137943AP | S, DREAMWISP NSL, MED CON, W/HGR, AP |
| 1137943CE | S, DREAMWISP NSL, MED CON, W/HGR, CE |
| 1137943RC | S, DREAMWISP NSL, MED CON, W/HGR, RC |
| 1137944AP | M, DREAMWISP NSL, MED CON, W/HGR, AP |
| 1137944CE | M, DREAMWISP NSL, MED CON, W/HGR, CE |
| 1137944RC | M, DREAMWISP NSL, MED CON, W/HGR, RC |
| 1137945AP | L, DREAMWISP NSL, MED CON, W/HGR, AP |
| 1137945CE | L, DREAMWISP NSL, MED CON, W/HGR, CE |
| 1137945RC | L, DREAMWISP NSL, MED CON, W/HGR, RC |
| 1137946AP | XL, DREAMWISP NSL, MED CON, W/HGR, AP |
| 1137946CE | XL, DREAMWISP NSL, MED CON, W/HGR, CE |
| 1137946RC | XL, DREAMWISP NSL, MED CON, W/HGR, RC |
| 1137916RTL | FITPK DREAMWISP NSL, MED CON W/HGR -RTL |

| | |
|---------|--|
| 1145055 | XS, Therapy Mask 3100 SP, w/oHGR, GBL |
| 1145056 | S, Therapy Mask 3100 SP, w/oHGR, GBL |
| 1145057 | M, Therapy Mask 3100 SP, w/oHGR, GBL |
| 1145058 | L, Therapy Mask 3100 SP, w/oHGR, GBL |
| 1145059 | MW, Therapy Mask 3100 SP, w/oHGR |
| 1145060 | Therapy Mask 3100 NC-SP, demopack |
| 1145061 | Therapy Msk 3100 NC, lab fitpack, single |
| 1145062 | Therapy Msk 3100 SP, lab fitpack, single |
| 1145063 | Therapy Mask 3100 NC-SP, lab combo |
| 1145064 | XS, Therapy Mask 3100 NC, cushion, RP |
| 1145065 | S, Therapy Mask 3100 NC, cushion, RP |
| 1145066 | M, Therapy Mask 3100 NC, cushion, RP |
| 1145067 | L, Therapy Mask 3100 NC, cushion, RP |
| 1145068 | MW, Therapy Mask 3100 NC, cushion, RP |
| 1145069 | XS, Therapy Mask 3100 SP, cushion, RP |
| 1145070 | S, Therapy Mask 3100 SP, cushion, RP |
| 1145071 | M, Therapy Mask 3100 SP, cushion, RP |
| 1145072 | L, Therapy Mask 3100 SP, cushion, RP |
| 1145073 | MW, Therapy Mask 3100 SP, cushion, RP |
| 1145074 | Therapy Mask 3100 NC-SP, frame, RP |
| 1145459 | XS, Therapy Mask 3100 NC, w/HGR, GBL |
| 1145460 | S, Therapy Mask 3100 NC, w/HGR, GBL |
| 1145461 | M, Therapy Mask 3100 NC, w/HGR, GBL |
| 1145462 | MW, Therapy Mask 3100 NC, w/HGR, GBL |
| 1145463 | L, Therapy Mask 3100 NC, w/HGR, GBL |
| 1145464 | XS, Therapy Mask 3100 SP, w/HGR, GBL |
| 1145465 | S, Therapy Mask 3100 SP, w/HGR, GBL |
| 1145466 | M, Therapy Mask 3100 SP, w/HGR, GBL |
| 1145467 | MW, Therapy Mask 3100 SP, w/HGR, GBL |
| 1145478 | L, Therapy Mask 3100 SP, w/HGR, GBL |
| 1145916 | Therapy Mask 3100 NC, lab fitpack, GBL |
| 1145917 | Therapy Mask 3100 SP, lab fitpack, GBL |