

Drägerwerk AG & Co. KGaA, 23542 Lübeck

To all customers of the:
Dräger Infinity ACS Critical Care Workstation (Evita Infinity V500)
Dräger Infinity ACS Neonatal Care Workstation (Babylog VN500)
Dräger Evita V300
with the optional PS500 power supply unit

**Subject: Update regarding the safety notice
issued for the PS500 external battery in January 2016**

January 2017

Dear Sir/Madam,

As part of our continued market and product monitoring we had become aware of cases in which the battery capacity of the optional PS500 power supply unit for the Infinity Critical Care Workstation (Evita Infinity V500), Infinity Neonatal Care Workstation (Babylog VN500) and Evita V300 was unexpectedly reduced. Dräger then consequently issued the above referenced safety notice in January 2016.

Dräger now announces that we have developed an improvement and are making it available to you free of charge so that you can fully utilize your device.

The device modifications include the following:

- Installation of a new software 2.50 with improved battery charge management
 - Improved charge management for external batteries
 - Option of performing a battery check to determine the battery's current runtime
 - Option of displaying the remaining battery capacity in minutes
- Installation of an additional internal backup battery

Drägerwerk AG & Co. KGaA
Moislinger Allee 53-55
23558 Lübeck, Germany
Postal address:
23542 Lübeck, Germany
Tel +49 451 882-0
Fax +49 451 882-2080
info@draeger.com
www.draeger.com
VAT no. DE135082211

Bank details:
Commerzbank AG, Lübeck
IBAN: DE95 2304 0022 0014 6795 00
Swift code: COBA DE FF 230
Sparkasse zu Lübeck
IBAN: DE15 2305 0101 0001 0711 17
Swift code: NOLADE21SPL

Registered office: Lübeck
Commercial register:
Local court Lübeck HRB 7903 HL
General partner: Drägerwerk Verwaltungs AG
Registered office: Lübeck
Commercial register:
Local court Lübeck HRB 7395 HL

Chairman of the Supervisory Board
for Drägerwerk AG & Co. KGaA
and Drägerwerk Verwaltungs AG:
Prof. Dr. Nikolaus Schweickart
Executive Board:
Stefan Dräger (Chairman)
Rainer Klug
Gert-Hartwig Lescow
Dr. Reiner Piske
Anton Schrofner


- Replacement of the power supply unit to support both the external battery and an additional internal battery
- Replacement of the existing external batteries

Please note that batteries are wear parts and the remaining maximum capacity not only depends on aging of the batteries themselves but also on the extent and the way it is used, the charging cycle behavior of the user and potential variations in quality during the manufacturing of the battery. The installation of a backup battery will ensure that the internal battery will continue to supply power should the external battery fail. The battery test, as described in the instructions for use, gives information about the remaining maximum battery capacity.

Your DrägerService representative will contact you in the coming weeks to arrange an appointment. Please continue to observe the safety notice issued for the PS500 external battery dated January 2016 until your devices have been updated with the improved system.

We would like to apologise for the inconvenience caused by the implementation of this technical solution, but consider the action as a necessary preventive measure to increase patient safety. Please feel free to contact your local Dräger representative if you have any questions.

Kind regards,

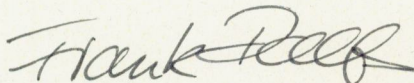

Frank Ralfs
Head of Product Management
Respiratory Care



Frank Clanzett
Head of Clinical and Regulatory
Affairs