

Advisory Notice

Reference Number	BES080217/2	Date of Issue	08.02.17
Device	Stealth Flip Down Stems	Range of Devices	TWB480-17FDM TWB486-17FDM TWB480-1FDM TWB486-1FDM

<input type="checkbox"/> Immediate Action	<input checked="" type="checkbox"/> Action
<input type="checkbox"/> Update	<input type="checkbox"/> Information

Problem
<p>The newest version of the flip down stems produced by Stealth Products Inc. (released May 2016) may have nuts and bolts within the stem which do not fully secure. Due to the two parts not fully securing the bolt can come loose. Indication of parts in Appendix A. The end of the bolt should be at least flush with the outside of the nut.</p> <p><u>Note:</u> Flip Down stems with the Button-release mechanism are not affected – only the flip down stems with the Collar-release mechanism are possibly affected (design of stem in diagram in Appendix A).</p>
Action by Whom
<p>BES to make contact with all customers who have purchased these stems since release. BES have checked all current stocks to ensure that no further at-risk stems are despatched.</p> <p>Stealth Products Inc. have confirmed that they have now changed the bolts that are being used in the manufacturing process to ensure that are secure.</p>
Action Required
<p>Customers to check that bolt end is at least flush with the outside of the nut on the stem. If the bolt is not flush with the nut or it is loose, BES will supply and fit new bolts and nuts for all that are identified.</p>

Distributed to
Customers who have purchased flip down stems since May 2016.
Contact
Nikki ter Haar – Quality Assurance Manager Tel: 01179 666 761 Email: nth@bescorporate.net

Appendix A

