



## Attachment A: Air in Kangaroo Joey and ePump Enteral Feed Sets

| Product Code | Product Description                  | Affected Lots   |
|--------------|--------------------------------------|---|
| 666064       | Joey 500ML Set FEED ONLY             | All product manufactured from April, 2022 with lot numbers 22D to July 2023 to lot numbers 23G. |
| 666106       | 666106 Joey 1000ML Set FEED ONLY     |   |
| 777501       | Joey ENPLUS Spike set FEED ONLY      |   |
| 777502       | Joey Dual ENPLUS Spike Set FEED ONLY |   |
| 777503       | Joey ENPLUS 3 in 1 FEED ONLY         |   |
| 777106       | EPUMP 1000ML EZ Bag FEED ONLY        |   |


### Check the feed set for entry of air in the tubing during priming.

- Before connecting to the patient:
  - Perform additional priming by pressing and holding the “HOLD TO PRIME FEED BUTTON.”
  - Check the tubing between the pump and the patient for air.
  - Do not use the feed set if a single segment of air or if several air bubbles cumulatively are > 1.3 cm or 0.5 inches.

### Check the feed set for entry of air while feeding.

- During patient assessment rounds:
  - Monitor frequently for excessive air in the tubing.
  - Discontinue use and discard the feed set if a single segment of air or if several air bubbles cumulatively are > 1.3 cm or 0.5 inches.

**Note:** It is not uncommon, and not a problem, to see a small amount of air in the feed tubing (up to 1.3 cm or 0.5 inches).

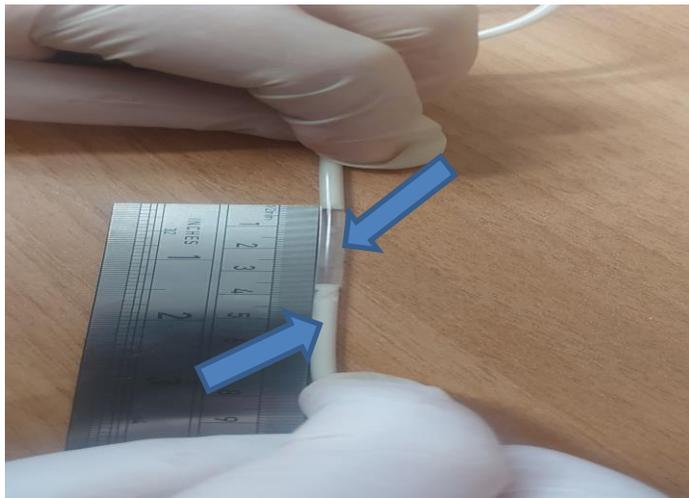

\*This is an example and the air in the tubing may not be as pronounced as shown in the image.