

**Urgent Field Safety
Notice**

14th January 2016

Dear Peritoneal Dialysis Patient,

Affected Product All HomeChoice Automated PD System and HomeChoice PRO Automated PD System devices

Product Code	Description
R5C8320	HomeChoice PRO Automated PD Cyclers, 230V

Actions taken by Baxter Baxter Healthcare Corporation is updating the HomeChoice Patient At-Home Guide Addendum to include a discussion of the expected operating sounds associated with all HomeChoice and HomeChoice PRO devices. The new labeling will address the following:

The HomeChoice cyclers utilize an air (pneumatic) pump; some sounds related to the operation of this pump are expected as part of normal operation of the HomeChoice cycler. There will be sounds like humming, swishing, clicking, and venting (air being released) that are normal for the operation of the cycler. In certain portions of the therapy, the sound level is expected to increase which is also part of normal operation. If there is a significant change in the sound level or a new, previously unheard sound when using the HomeChoice cycler, please contact your doctor and/or nurse or Baxter Technical Services.

Action to be taken by the user Baxter is kindly asking that you take the following actions:

- Please keep this letter with your Patient At-Home Guide.

Further information and support For general questions regarding this communication, contact Baxter at 01 206 5500, between the hours of 9am – 5pm

We apologize for any inconvenience this may cause you.

Baxter

Yours Sincerely,



Ian Gavigan
Head of CQA UK/Ireland
Baxter Healthcare Ltd.
Deansgrange Business Park
Blackrock
Co. Dublin
Ph: 01 2065500