

Instructions of Removing the Joystick Knob



WARNING

- If the joystick is not at its neutral position, all the LED's will flash slowly when turning on the controller. For your safety, the wheelchair will not operate. Do not use this powered wheelchair and contact your local dealer to adjust the attendant controller.
- If the joystick is not at its neutral position after releasing it, the wheelchair could continue moving on their own. For your safety, do not operate this powered wheelchair and contact your local dealer to adjust the attendant controller.

The following steps describe how to remove the joystick knob if it is necessary to adjust the neutral position of the attendant controller.




1. Turn off the power before removing the joystick knob.



- 2.1. Press the switch in order to remove the controller.



- 2.2. Keep pressing the switch and pull the controller backwards and proceed to step 4.

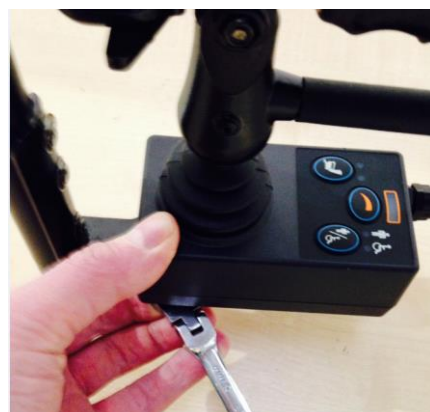

- 3.1. If the controller is the dual control model, Loosen the four screws under it.



32. Loosen screws and remove the controller. Remove ball and replace dual unit.



4. Remove the controller from the attendant controller bracket.



5. Remove the joystick knob.



6. Press the control lever and insert the joystick lever into the hole of control lever.



7. Slide the controller into the attendant controller bracket.



8. Check the neutral position: If the joystick is not at its neutral position, all the LED's will flash slowly when turning on the controller.

National agent:

Sold and serviced by:

KARMA MEDICAL PRODUCTS CO., LTD.

Http://www.KarmaMedical.com

E-mail:globalsales@karma.com.tw

KARMA reserves the right to modify information herein without further notice.

Karma reserves the right to modify information herein without further notice.

Rev.00 2014.12