

Customer Information Bulletin

Part Numbers	Product Description	Issued by	Hellie King
100-172S	Syringe extension set	Document No	CB15-006, Rev 02
100-172SLL	Syringe extension set	Date	21/12/15

Purpose of Bulletin

CME Medical would like to inform our customers of the planned replacement of our current syringe extensions sets PN100-172S and PN100-172SLL with a new version.

The current extension set is predominantly used with our T34™ and T60™ ambulatory syringe pumps.

From week commencing 14/12/15 all current orders will be satisfied with the substitute administration set PN30-7100.


Detail

The current administration set part numbers are **100-172S** or **100-172SLL** and will be substituted with **30-7100**

The differences between the administration sets are detailed below:

Part Number	100-172S / 100-172SLL	30-7100
Set composition	Non-DEHP PVC plasticized with TOTM, BSE-TSE free, latex free and EVA free anti-kink tubing	PE/PVC Infusion Lines with flexible, transparent PVC outer layer and internal Polyethylene, NON DEHP, Latex free, anti-kink tubing
Set length	100cm	100cm
Priming volume	0.5mL	0.8mL
Additional features	ASV/ anti reflux Male luer (patient end), female luer (syringe connector)	Rotating male luer (patient end), ASV Female luer (syringe connector), slide clamp

Current set PN100-172S/100-172SLL

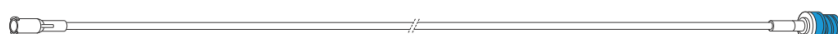

Subtitute Set 30-7100



NOTE:

The substitute set can only be used with the T34 in the lockbox with syringe sizes up to 20mL. The female luer/ASV attached to a 30mL syringe tip prevents the lockbox lid closing.

The most significant change on PN30-7100 is the male rotating luer lock that attaches to the patients access device.

Please follow the advice for correct connection of the male rotating luer to patient access device:

1. *Insert Male luer firmly into access device*



2. *Once in position rotate one quarter turn clockwise*



3. *Secure in place using the rotating locking collar as shown*



If you have any further questions or enquiries, please contact Customer Support on 01253 206700 or email customersupport@cmemmedical.co.uk.