Ár dTagairt: MNE/19-191

Bobigny, 8 Eanáir 2020

LITIR CHLÁRAITHE LE HAITHÉANTAS DÁ FÁIL – Leanann an post seo ar phost roimhe seo.

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<i>Faisnéis faoin sruth IFU (IN121 V06-2018):</i>	<i>Faisnéis mhodnaithe san IFU:</i>
<p>INDICATIONS OF USE</p> <p>PETERS SURGICAL instruments are solely designed for laparoscopic surgery. Depending on each instrument and its model, they may be employed for:</p> <ul style="list-style-type: none"> <li>- mechanical freeze incision,</li> <li>- monopolar incision via high-frequency (HF) current,</li> <li>- monopolar coagulation via high-frequency (HF) current,</li> <li>- mechanical dissection,</li> <li>- atraumatic or traumatic prehension of tissues</li> </ul> <p>Maximum assigned voltage: 5,000 at peak voltage (the generator should be set at a lower voltage).</p> <p>Such instruments should only be handled by qualified practitioners having gained due experience thereof.</p> <p>Make sure that the voltage assigned to any accessory used in combination with a HF generator is superior to that of the generator itself (power setting, mode); please refer to IEC 60601-2-2 requirement. Please also refer to the Instructions' Manual of the HF generator.</p> 	<p>PETERS SURGICAL instruments are solely designed for laparoscopic surgery. Depending on each instrument and its model, they may be employed for:</p> <ul style="list-style-type: none"> <li>- mechanical freeze incision,</li> <li>- monopolar incision via high-frequency (HF) current,</li> <li>- monopolar coagulation via high-frequency (HF) current,</li> <li>- mechanical dissection,</li> <li>- atraumatic or traumatic prehension of tissues</li> </ul> <p>Maximum assigned voltage: 4,000 at peak voltage (the generator should be set at a lower voltage).</p> <p>Such instruments should only be handled by qualified practitioners having gained due experience thereof.</p> <p>Ensure compatibility with all other associated instruments; in particular, the connections and voltage supplied by the HF generator should be lower than the assigned voltages of the accessories being used in combination (see IEC 60601-2-2). Check the instruction for use of the cable and of the HF generator regarding its operating, as well as its power and mode settings.</p> <p>NE: Only surgical monopolar cables may be connected to PETERS SURGICAL monopolar instruments</p> 

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