



## **Field Safety Notice**

### **Spectra Plus Powered Wheelchair** **Recommended Safe Slope Correction**

#### **Important Safety Information:**

In the interests of the safety of our customers we would like to bring your attention to a minor correction that has been made to the Spectra Plus user manual.

#### **Correction Details:**

The statement; "Take care when using your power chair on slopes of up to 1 in 5 (10% up to 20% gradient)", has been amended to read; "The recommended safe slope for the wheelchair is advised as 6 degrees (1 in 10 gradient), when the chair is configured to the recommended driving position (seat angle at 6 degrees, i.e. fully lowered, and backrest angle at 12 degrees, i.e. fully raised). This is the most stable set-up for driving your wheelchair. Any alternative seating or backrest position will adversely affect the stability of your wheelchair".

Should you require a copy of the amended user manual, or any additional information / clarification please contact Invacare directly, via the Technical Services Department who will be pleased to assist with any enquiry.

The latest issue of the Spectra Plus user manual is also available for viewing / downloading via the Company's website ([www.invacare.co.uk](http://www.invacare.co.uk)).

#### **Technical Services Department**

Tel. 01656 647327

Fax. 01656 649016