



# Care & Independence Systems Ltd.

Independence House, Field Farm Business Centre, Launton, Oxon. OX26 5EL

Tel: 01869 278910 Fax: 01869 278533

email: [sales@careandindependence.co.uk](mailto:sales@careandindependence.co.uk)

[www.disabilityequipment.com](http://www.disabilityequipment.com)

www.disabilityequipment.com

## Urgent Field Safety Notice

**Date:** January 2008

**Devices affected:** InChair sling - Instructions for use

**Action to take:** Following a detailed review of the instructions for use it was felt that more emphasis on the importance of using the safety pommel should be made. If the safety pommel is not used whilst a person is being lifted there is a potential danger that they could slide and fall through the front of the sling resulting in serious injury.

The new instructions (Issue 2) are attached to this notice.

No change has been made to the sling design or the way that it is used.

It is recommended that prescribers and equipment providers ensure that all affected users and/or their carers receive a copy of the new instructions for use and are instructed to read it and follow its guidance. When you have read this notice please pass it on to all those that need to be aware of it within your organisation to ensure that the information cascades to everybody affected.

**Contact reference person:** Richard Handley  
Managing Director  
Care and independence Systems Ltd  
Field Farm Business Centre  
Launton  
Oxfordshire  
OX26 5EL

Telephone 01869 278910  
Email [Richard@careandindependence.co.uk](mailto:Richard@careandindependence.co.uk)

Signature.....



Directors: P. Cheal, R. Handley. Secretary: D. J. Cheal LL B. M.A.

Registered Eng/Wales No. 4214056

VAT No. 781471809

Reg. Office: Unit 1A, Hightown Industrial Estate, Crow Arch Lane, Ringwood, Hants. BH24 1ND

Member of the Spectra Care Group