

Urgent Field Safety Notice

Trade name / brand name:

Medi-tec Hoist

Date: 28/06/2008**Type of action required:**

Corrective Action

Details of products effected:

Medi-tec hoists

Descripton of problem:

The spreader / sling support bar has been incorrectly adjusted causing excessive ware on the pivot bolt.

This may result in the spreader / sling support bar becoming detached from the hoist.

The spreader / sling bar attachment has been re-designed to overcome this problem and replacement fittings are available from the manufacturers.

This notice needs to be passed on to all those who need to be aware within your organisation pr to and organisation where the potentially affected devices have been transferred.

Please transfer this notice to other organisations on which this action has an impact.

Please maintain awareness on this notice and resulting action for an appropriate period to ensure effectiveness of the corrective action.

Contact:

Aidservice Limited
Unit 15
Bradley Hall Trading Estate
Standish
Wigan
WN6 0XQ

Tel: 01257 425538

Fax: 01257 472181

Email: sales@aidservice.co.uk

Signature:

Date: