

---

# Axminster Respiration Monitor

Service Update 010608

## Action Required

---

SERIAL NUMBERS 001360 to 001660

# Axminster

Baby Sleepsafe

Monitor Model AE 010 M



---

## **IMPORTANT INFORMATION**

---

**It has been brought to our attention that if the monitor is subjected to physical impact the battery terminals can become loose. We recommend that the condition of the battery terminals be regularly inspected.**

**The problem has since been prevented with a spring coil battery terminal on monitors serial number 001660 to 001950 and by using our original battery terminal from 001951 to date. We recommend that if the monitor has a serial number between 001360 and 001660 and is not fitted with coil spring battery terminals that they are returned for upgrading. Serial numbers below 001360 use a flat spring made from a more resilient material which we have not experienced problems with, this has been used again from serial number 001951 to date.**

---

### **Recommendations for Care of All battery Terminals**

---

#### **Inspection**

---

At least every 12 months inspect the battery terminals for signs of corrosion and ensure they are applying firm pressure to the batteries. Replace corroded or deformed battery terminals.

#### **When Monitor is in Transit**

---

If the monitor is transported in such a way that it may become subject to impact, for example in the post or via a courier, please ensure that the batteries are removed.


## Upgrading

---

(For monitors with serial numbers between 001360 and 001660)



Replace terminals of this type.

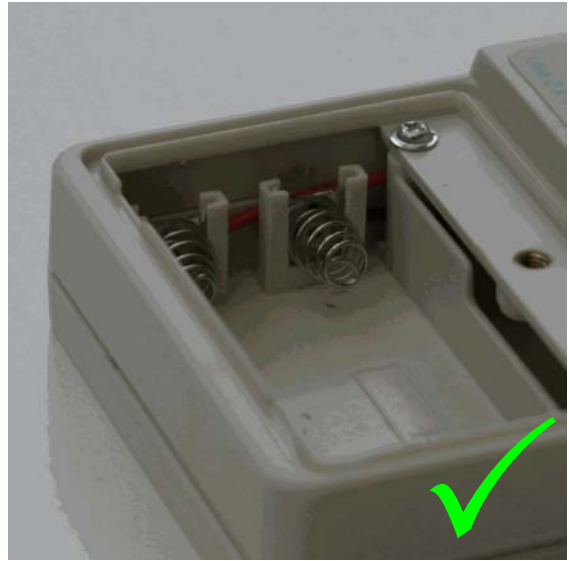

Coil Spring terminal fitted.

If your monitor(s) have serial number(s) between 001360 and 001660 and are not already fitted with coil spring battery terminals you may return the monitor(s) for upgrade. Please return to: -

Axminster Electronics Ltd  
Millwey Rise Industrial Estate  
Weycroft Avenue  
Axminster  
Devon  
EX13 5HU