

Field Safety Corrective Action Notice

Patterson Walking Frames – All models supplied between January and March 2013



Batch codes DMW12013 to DMW 82013


As a precautionary measure and in accordance with the information within our instructions for use can you please ensure you make the following checks:-



Ensure the 2 screws at the top are tight and free from sharp edges



Ensure the 2 screws at the bottom back are tight and free from sharp edges

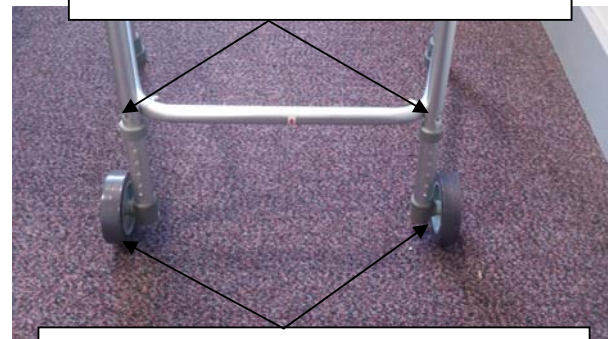

Ensure the 2 screws at the bottom front are tight and free from sharp edges

Wheeled Models

Ensure the 2 wheels are not loose and point straight ahead

If during these checks you find the above issues or any other fault with the walker please contact our customer services on 0844 124 330 for advice

Field Safety Corrective Action Notice

**Patterson Walking Frames – All models supplied between January and
March 2013**

Reply Letter

I can confirm we have checked the product and codes described in the FSCA and have none of these items / have checked (delete as applicable) as requested.

Hospital / Customer name

Account number

Name of person responding.....

Position.....

Date.....

Please complete this form and return to Customer services at Patterson Medical by email retail.sales@pattersonmedical.com or fax back 08448 730 100