

FIELD SAFETY NOTICE

Date: 2010-02-02
Author: Trina Turcotte, tturcotte@bhm-medical.com
Subject: FSN01/2010/ca.EN
Product group: Floor Lifts
Document number: FSN01/2010/ca.EN
Issue: 1
Case No: NA

SINGLE INSULATION POWER CABLE

SCOPE

This notice pertains to the following products manufactured by BHM Medical between May 2004 and March 2006 which are equipped with an internal battery charger:

Part name	Part number
CRICKET 3	83301003, 83301005, 83301007.20
CYPRESS-2	81501003, 81521003, 81541005
ERGOLIFT	86000003, 86000008
ERGOLIFT 600	86100005
ERGOSTAND	86400008
EXPLORER	86000005.14
EXPLORER 600	86100005.14
GRADUATE	82501004, 82501005.09, 82601003, 82601004, 82601008
JUNIOR	82001044, 82001045, 82001045.09, 82001047.20, 82101043, 82101044, 82101048, 82121043, 82121045, 82141045
MEDILIFTER-IV	87000003
MEZZO	87001003.01
MINISTAND	86500004, 86500005, 86500007.20, 86501003, 86501003.01, 86501004, 86501007.20, 86601003, 86601004, 86601008
ML-4	87001008
PRIMO	87000003.01
SHERPA	83301005.14
SSL-2	84001003, 84001008
STANDUP	86500005.09, 86500035.09, 86501005.09, 86501035.09
TREKKER	86400005.14
XTREME	87001005.09

BACKGROUND

During the assembly of some of the above mentioned products, an improper power cable was installed on the control box. This power cable is single insulated, which increases the risk of premature failure.


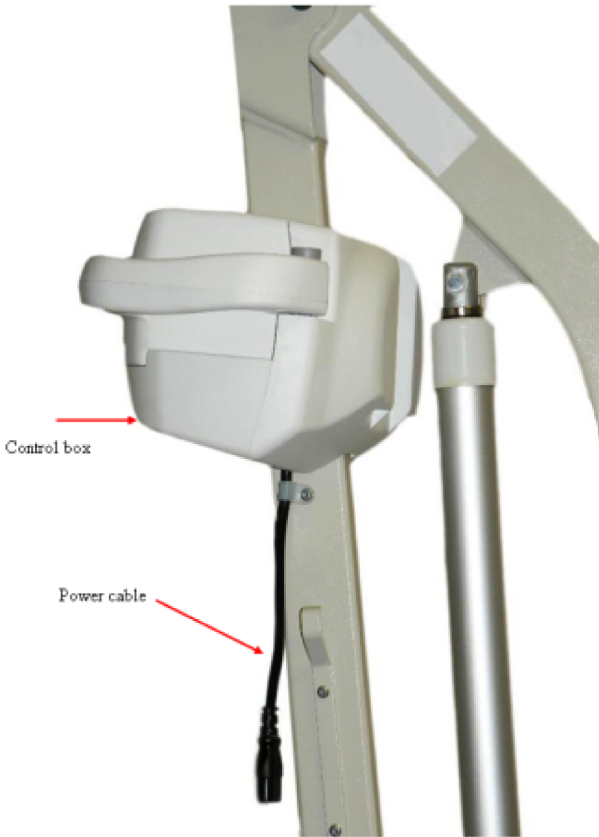


Fig 1: Power cable location

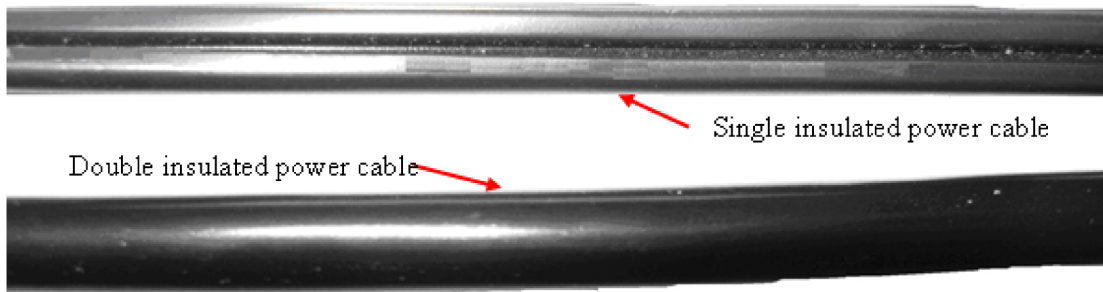

HEALTH HAZARD ANALYSIS

A power cable failure could result in a connection between the metal structure of the hoist and the main power. Therefore, there is a risk of electrical shock for the user while the hoist is plugged to an electrical outlet.

ACTIONS TO TAKE

1. Make sure that you carefully read and understand the FSN and the FSN Receipt form.
2. Make sure that this notice (including the FSN Receipt form) is distributed to all those who need to be aware within your organization or to any other organization to which the potentially affected hoist might have been transferred.
3. Inspect your hoist to determine if it is equipped with the single insulated power cable. The improper power cable can be located and identified by visual inspection (refer to Fig. 1 and Fig. 2).
4. If your hoist is equipped with a single insulated power cable, stop using it and call your dealers/distributors for a power cable replacement.
5. Complete and return the attached FSN Receipt form to your dealers/distributors or to BHM Medical **within 10 days** following reception.

NEXT STEP

The single insulated power cable will be replaced, free of charge, by our dealers/distributors.

If you have any further questions or special needs, please contact your dealers/distributors or our service department at:

BHM Medical Inc.

2001 Tanguay, Magog, QC, Canada J1X 5Y5

Telephone: (819) 868-0441

Fax: (819) 868-2249

E-mail: service@bhm-medical.com