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Serial numbers of affected devices and part

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There was an incident on 1 chair from the totally sold 5754 pcs C4 therapy chair. No injury happened. The pivot bolt broken in the chair. 19 pcs of pivot bolt broken at all. All the 19 pcs on the UK market. The shearing probability is very low. Shearing can happen only when more risk factors can act together: overweight patients, misuse (chair crash into the wall), usage of the chair more than the suggested lifetime, continuous 3 shifts operation, safety distances are not kept, if the serial nr of the chair is in the affected chairs list etc.

The breakage did not cause any personal injuries and the tests verify that even mild injuries may happen only with physically handicapped patients.

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Our suggestions:

Nr. 1

- If the chair is C4 model and younger than 09/22/2008
– **No action needed.**

- If the chair is C4 model and older than 09/22/2008 but there is no 3 shift operation and no frequent occurrence of overweight patient (more than 135 kg) and If the chair not used for patient transport and If the chair can not crash into wall (misuse).

– **No action needed.**

Nr. 2

- If the chair is older than 22/09/2008 and
- The serial nr is on the attached list and
- There is 3 shift operation and
- frequent occurrence of overweight patient (more than 135 kg).

– **Change the screw to 10.9** 

Nr. 3

- If the chair used for patient transport and
- If the chair can crash into wall (misuse) and
- If the nr 2 points risks occur

– **Repair the chair with repairing KIT** 

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This notice needs to be passed on all those who need to be aware within your organization or to any organization where the potentially affected devices have been transferred. (If appropriate)

Please transfer this notice to other organizations on which this action has an impact. (If appropriate)

Please maintain awareness on this notice and resulting action for an appropriate period to ensure effectiveness of the corrective action. (if appropriate)

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