


How to monitor the on-body injector

- ❖ For the first 27 hours you are wearing the on-body injector, you should see a slow flashing green light. Two minutes before dose delivery begins, you will hear a series of beeps.
 - ❖ When dose delivery begins, you will see a fast green light.
 - ❖ The dose will take 45 minutes to deliver.
 - ❖ For the next 45 minutes, monitor the on-body injector frequently for leaks during dose delivery.
 - ❖ When the delivery is done, you will hear a long beeping sound and the status light will be solid green.
 - ❖ Now, slowly pull off the on-body injector. Check to see if it is empty.
- Call your healthcare professional immediately if you have any issues with the on-body injector, or symptoms of infection such as fever, chills or sweating.
 - Do not try to secure the on-body injector if it becomes loose or try to reapply it.

Call your healthcare professional immediately if any of the following happens - you may not have received your full dose:

- The status light flashes red.

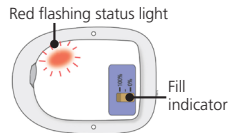


- The adhesive (the part touching your skin) is clearly wet and/or there are drips.



- The adhesive comes loose from the skin at any time before the end of dose delivery.



- If there is no line in the fill indicator next to 0% after the 45-minute dose delivery period is over.

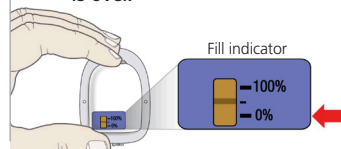

DO NOT try to secure the on-body injector if it becomes loose or reapply it under any circumstances. Call your healthcare professional immediately as you may need a replacement dose.

Neulasta® (pegfilgrastim)

On-body Injector Patient Alert Card

- ❖ Please open this card and read both sides.
- ❖ Keep it with you.
- ❖ This card contains important safety information that you need when receiving treatment with the pegfilgrastim on-body injector.
- ❖ Show this card to your healthcare professional if you need to seek urgent medical advice for the on-body injector.

Important information

The purpose of this card is to help prevent medication errors - mainly, drug underdose.

- ❖ A drug underdose may happen if the on-body injector comes loose from your skin, leaks, or otherwise malfunctions.
- ❖ If that happens, you may be at increased risk of a serious infection.

For more information:

- ❖ Read the Package Leaflet that comes with your on-body injector.
- ❖ The Package Leaflet is also available at <http://www.medicines.ie/>

The following activities may interfere with the on-body injector's performance:

- ❖ Keep the on-body-injector dry for the last 3 hours prior to start of dose delivery.
- ❖ **Do not** knock or pull the on-body injector.
- ❖ **Do not** get lotions, creams, oils or cleaning agents near the on-body injector.
- ❖ **Do not** sleep on the on-body injector or apply pressure during wear especially during dose delivery.
- ❖ **Do not** expose the on-body injector to CT scan, MRI, x-ray, ultrasound, radiation treatment, or hyperbaric chambers.
- ❖ **Do** wear loose clothing and be careful when changing clothes.
- ❖ **Do** keep the on-body injector at least 10 cm (4 inches) away from electrical equipment (such as mobile phones or microwaves).

Please read this card and keep it with you

This information should be completed by your healthcare professional

Your on-body injector was applied to your skin:

Day Time AM / PM

Your dose delivery will start around:

Day Time AM / PM

On-body injector lot number:

Emergency contact information

Name and contact number of healthcare professional:

Surname, Given Name

Contact Number

Out-of-hours urgent care contact details:

If you get any side effects, talk to your doctor, pharmacist or nurse. You can also report side effects directly via HPRC Pharmacovigilance, Earlsfort Terrace, IRL - Dublin 2; Tel: +353 1 6764971; Fax: +353 1 6762517. Website: www.hpra.ie; E-mail: medsafety@hpra.ie.