

## Medtronic Ireland Limited

T Block 3090-3094  
Lake Drive  
Citywest Business Campus  
Dublin  
DN24 XN47  
Ireland  
Tel: 01 511 1400  
Fax: 01 807 7220  
www.medtronic.ie

## URGENT FIELD SAFETY NOTICE

### Esoflip™ dilation catheter 30mm - Model # ES-330

#### Recall

April 2024

Medtronic Reference: FA1413

EU Manufacturer Single Registration Number (SRN): US-MF-000028763

Dear Healthcare Provider/Risk Manager,

The purpose of this letter is to advise that Medtronic is initiating a recall of the Esoflip™ dilation catheter 30mm - Model # ES-330, for the specific lot numbers listed below.

#### Issue Description:

Medtronic has determined that devices within specific lot numbers of Esoflip™ dilation catheter 30mm - Model # ES-330 may have non-conformances in saline conductivity that could negatively impact the performance of the product as indicated. If such devices are used, inaccurate esophageal diameter measurements may lead to these harms: delay to treatment, pain, bleeding, esophageal laceration, esophageal perforation, or respiratory distress.

From 01 April 2022 through 31 March 2024, there were 167 complaints related to catheters with low saline conductivity.

#### Product Scope:

Brand Name	Model Number	GTIN Number	Affected Lot Number
Esoflip™ dilation catheter 30mm	ES-330	10884521809451	22K0854JZ, 23H0552JZ
		20884521809458	22E0781JZ, 22E0997JZ, 22I0422JZ, 22I0424JZ, 22I0425JZ, 22J1362JZ, 22K0743JZ, 22K0744JZ, 22K0745JZ, 22K0746JZ, 22K0854JZ, 23A0109JZ, 23A0111JZ, 23A1200JZ, 23A1201JZ, 23A1202JZ, 23C0144JZ, 23C0668JZ, 23C0669JZ, 23D0777JZ, 23D0778JZ, 23H0047JZ, 23H0552JZ, 23H0553JZ, 23H0554JZ, 23H0623JZ, 23H0624JZ, 23L0331JZ, 23L0338JZ, 23L0339JZ, 24A0014JZ, 24A0015JZ, 24B0392JZ.

# Medtronic

## Medtronic Ireland Limited

T Block 3090-3094  
Lake Drive  
Citywest Business Campus  
Dublin  
DN24 XN47  
Ireland  
Tel: 01 511 1400  
Fax: 01 807 7220  
www.medtronic.ie

### Customer Actions:

- Immediately identify and quarantine all unused Esoflip™ dilation catheter 30mm - Model # ES-330 from the affected lot numbers.
- Return all unused affected product(s) in your inventory to Medtronic.
- Please complete and return the enclosed Customer Acknowledgement Form, even if you have no product to return.
- If purchased from a distributor, contact your distributor directly to arrange for the return of the product back to your distributor.
- This notice needs to be passed on to all those who need to be aware within your organization or to any organization where the potentially affected devices have been transferred.
- Please transfer this notice to other organizations on which this action has an impact.
- Please maintain a copy of this notice in your records.

### Patient Recommendations:

There are no additional actions required for patients where an Esoflip™ dilation catheter 30mm - Model # ES-330 was used during a procedure. These patients should continue to be monitored in accordance with your medical facility's standard care protocols.

### Additional Information:

Medtronic has notified the Competent Authority of your country of this action.

We regret any inconvenience this may cause. We are committed to patient safety and appreciate your prompt attention to this matter. If you have any questions regarding this communication, please contact your local Medtronic representative or via Tel. No: 01 511 1400

Sincerely,



Natasha Mthethwa  
Senior Regulatory Affairs Specialist

Enclosed:

Customer Acknowledgement Form