**Dringende Sicherheitsinformation / Produkt Rückruf**  
**Urgent Field Safety Notice / Recall**

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**Zu Händen / For the attention of**

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Customers who received one or more of the products listed below.

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All products delivered by Otto Bock SE & Co. KGaA in the period from May 1<sup>st</sup> 2017 to January 19<sup>th</sup>, 2018, are affected.

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The PushValve / Mag Valve is a threadless one-way valve. It is used to evacuate air from the prosthetic socket. For valves that have been delivered during the specified period it has been found that, under certain conditions, the white valve-insert may fall out.

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Out of an abundance of caution, Ottobock SE & Co KGaA has decided to voluntarily provide a replacement.

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The cause of the fault is that some of the installed valve-inserts have a diameter outside the tolerance range. As a result, the interference fit assembly may not perform as intended. Since the valve-insert is smaller than the specification, there is play between the valve and the housing, which can lead to the valve falling out of the housing in the event of strong overpressure.

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If this fault occurs, it can lead to a loosening of the socket and an increased risk for the patient. Although we have not been aware of any case in which the occurrence of the failure has resulted in injury or serious deterioration of a user's health, we seek to replace any product on the market that may be affected.

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The failure only concerns the upper part. The Lamination Ring is not affected and can remain in the socket.

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Product delivered before May 1, 2017 and after January 19, 2018 are not affected.

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Please forward the patient letter you received from us immediately to all patients who have been provided with products from the specified period.

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Please send the replacement valves that you are going to receive from us in quarter 1, 2019 together with the Information for the User (Patient Letter) promptly to all affected users. The replacement can be carried out independently by the patient.

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Products for replacement will be provided by Otto Bock.

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Sign the attached acknowledgement of receipt (Customer Reply Form) and send it back to your local Otto Bock Partner. Use therefor the Fax number or E-Mail address known by you.

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Incurred costs will be managed in a separate step.

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This letter must be forwarded to all persons in your company who should be informed about those facts.

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Please transfer this notice to all other organisations on which this corrective action has an impact.

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Please translate this notice and also the reply in your national language on request.

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A copy of this “Urgent Field Safety Notice” has been forwarded to the German authority Bundesinstitut für Arzneimittel und Medizinprodukte (BfArM, German Federal Institute for Drugs and Medical Devices).

**Kontaktperson/ Contact reference person:****Dominik Schmidt**

Head of Regulatory Affairs | Safety Officer Medical Devices

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Max-Näder-Straße 15 | D-37115 Duderstadt

T +49 (0) 5527 848 1674 | F +49 (0) 5527 848 81674

[Dominik.Schmidt@ottobock.de](mailto:Dominik.Schmidt@ottobock.de) | [www.ottobock.de](http://www.ottobock.de)