
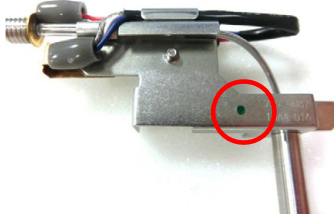
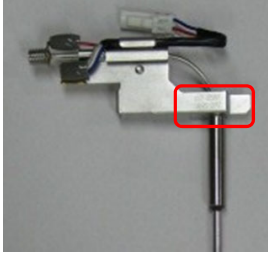



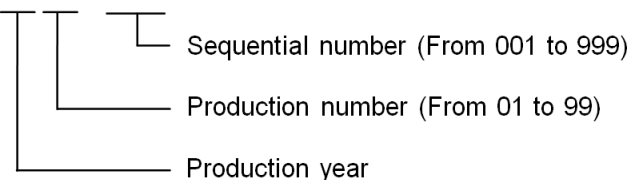
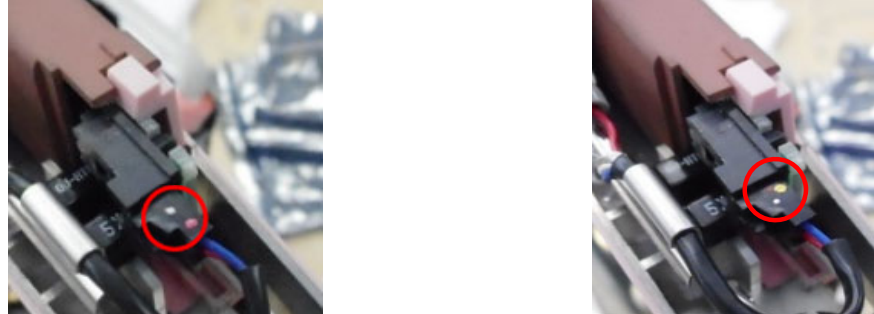
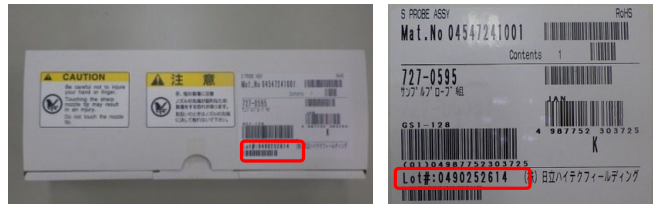
Target

With this procedure potentially affected Sample Probes can be identified both already installed Sample Probes and still boxed Sample Probes.

Identification of potentially affected sample probes for cobas c501/ cobas c502 and cobas c311

There are two HHT production plants NAKA & OMUTA producing Sample Probes. Potentially affected Sample Probes can be easily encountered by the following procedure. If a yellow dot is applied to the Sample Probe connector the Sample Probe is already equipped with a modified connector.


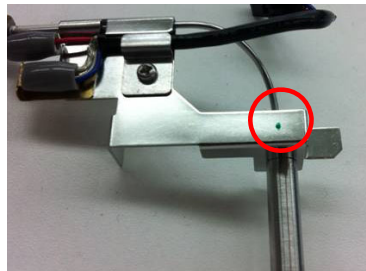


04547241001 PROBE SAMPLE (cobas c501/ cobas c502)	
05899427001 PROBE SAMPLE S (cobas c311)	
Affected serial number which printed on the probe	<p>NAKA; 1509-001 onward, and <u>no marked yellow dot</u> on the connector. OMUTA; 1535-001 onward, and <u>no marked yellow dot</u> on the connector.</p> <p>Refer to Place of yellow dot on the connector in this table for detail.</p>
Identification of production site in HHT	<div style="display: flex; justify-content: space-around;">   </div> <p>Either NAKA or OMUTA is indicated on the original package. Either Black or Green dot is marked on the sample probe.</p> <p>Black dot: Produced by NAKA Green dot: Produced by OMUTA</p>
Printed place on sample probe	<div style="display: flex; justify-content: space-around;">   </div> <p>Refer to Rule of printed serial number for detail.</p>

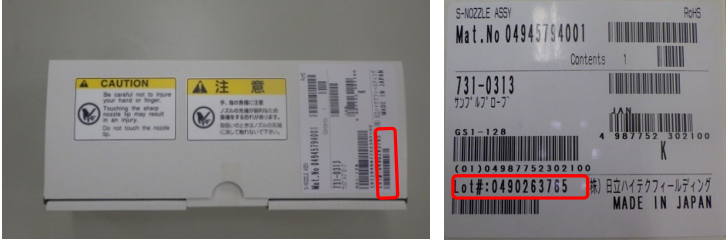
<p>Rule of printed serial number</p>	<p>727-0587 ← Part number YY AA – BBB ← Serial number</p>  <p>Sequential number (From 001 to 999) Production number (From 01 to 99) Production year</p> <p>Serial number printed on the probes for cobas c501/c502 and cobas c311 are making a common numbering by HHT's two production lines.</p>
<p>Place of yellow dot on the connector</p>	 <p><u>No yellow dot mark</u> on the connector is affected. If the yellow mark on the connector is existed, the probe was modified done by HHT despite if the Serial number which printed on the probe was met.</p>
<p>SAP lot number which printed on the original package</p>	
<p>Affected Sample Probe for cobas c311 (P/N 724-0920)</p>	<p>Lot number which indicated on the logistics label 0490232027, 0490233855, 0490236427, 0490238543, 0490241937, 0490248322, 0490250880, 0490235713, 0490252029, 0490256963 (not affected), 0490254335 (new)</p>

<p>Affected Sample Probe for cobas c 501 and c502 (P/N 727-0595)</p>	<p>0490233762, 0490233764, 0490233766, 0490233767, 0490233777, 0490235904, 0490235905, 0490235906, 0490235907, 0490235949, 0490235950, 0490237594, 0490237595, 0490237353, 0490237354, 0490237597, 0490237598, 0490237600, 0490237602, 0490237604, 0490237606, 0490238955, 0490238957, 0490238893, 0490238895, 0490238896, 0490238898, 0490238899, 0490242149, 0490242151, 0490242153, 0490242155, 0490242327, 0490242328, 0490242329, 0490242330, 0490242331, 0490250491, 0490250492, 0490250493, 0490250494, 0490250495, 0490250496, 0490251032, 0490251033, 0490251034, 0490251036, 0490252524, 0490252527, 0490252529, 0490252532, 0490252247, 0490252249, 0490252251, 0490252252, 0490252272, 0490252398, 0490254550, 0490254551, 0490254552, 0490254553, 0490254554, 0490254555, 0490254804, 0490254805, 0490254806, 0490254807, 0490256999, 0490257000, 0490256993, 0490256994, 0490256995, 0490256996, 0490256997, 0490256998, 0490254808, 0490257001, 0490257847, 0490257848, 0490257849, 0490257850, 0490257851, 0490258652</p>
---	---

Identification of potentially affected sample probes for cobas c701/ cobas c702

HHT production plants NAKA & OMUTA producing Sample Probes as well for **cobas** c701/ **cobas** c702 and ISE. Potentially affected Sample Probes can be encountered by the following procedure.

<p>04945794001 SAMPLE PROBE (cobas c701/ cobas c702)</p>	
<p>Affected serial number which printed on the probe</p>	<p>ISE and Sample Probe for cobas c701/ cobas c702 do not have serial number printed on the probe itself.</p>
<p>Identification of production site in HHT</p>	<div style="display: flex; justify-content: space-around;">   </div> <p>Either NAKA or OMUTA is indicated on the original package. Either Black or Green dot is marked on the sample probe.</p> <p>Black dot : Produced by NAKA Green dot : Produced by OMUTA</p>
<p>Place of yellow dot on the connector</p>	<div style="display: flex; justify-content: space-around;">   </div> <p><u>No yellow dot mark</u> on the connector is affected. If the yellow mark on the connector is existed, the Sample Probe was already modified by HHT.</p>

<p>SAP lot number which printed on the original package</p>	 <p>Refer to the separate attachment for detail of SAP lot numbers which printed on the original package.</p>
<p>Affected Sample Probe for cobas c701/cobas c702 (P/N 731-0313)</p>	<p>0490238960, 0490238961, 0490238961, 0490238961, 0490238963, 0490242332, 0490251040, 0490251041, 0490252546, 0490252547, 0490252547, 0490254810, 0490254811, 0490221658, 0490223228, 0490223229, 0490225255, 0490226172, 0490226173, 0490226174, 0490227626, 0490227627, 0490227628, 0490230276, 0490230277, 0490230279, 0490232203, 0490232204, 0490232205, 0490232206, 0490233778, 0490233779, 0490233780, 0490233781, 0490233783, 0490235909, 0490235910, 0490235911, 0490235912, 0490235913, 0490237365, 0490237366, 0490237367, 0490237368, 0490237369, 0490237370, 0490238901, 0490238902, 0490238903, 0490238904, 0490242160, 0490242161, 0490242163, 0490242165, 0490242166, 0490250498, 0490250499, 0490250500, 0490250501, 0490250502, 0490252308, 0490252312, 0490252318, 0490252320, 0490252322, 0490254558, 0490254559, 0490254560, 0490254561, 0490254562, 0490257035, 0490257036, 0490257037, 0490257038, 0490257039, 0490257854, 0490257855, 0490257856, 0490257857, 0490257858</p>

<end of the document>